EGGEN  
DENTAL

## INSTRUCTION FOR USE

### DENTAL X-RAY FILM HOLDER

OCTOBER 9, 2023

ED-TF-PR-09-B

**Important Note:** This IFU (Instruction For Use) is intended to provide guidance on the proper usage of Dental X-Ray Film Holder that is produced by EGGEN DENTAL. It is essential to read and understand the instructions completely before using the product to ensure accurate performance and safe usage.

**Introduction:**

Thank you for choosing the EGGEN DENTAL, Dental X-Ray Film Holder. This innovative product is designed to assist dental professionals in obtaining accurate and high-quality dental X-ray images. This IFU provides comprehensive information on the correct usage, handling, and maintenance of the dental X-ray film holder.

**Product Description:**

Eggen-bite is a dental X-ray film holder intended to position and hold X-ray film inside the mouth when taking X-ray pictures of teeth. It is intended to assist dental professionals in obtaining clear and accurate X-ray images of a patient's teeth and oral structures.

This device is intended for use by trained dental professionals, such as dentists, dental hygienists, or radiographers, who are experienced in dental radiography techniques. It should be used in conjunction with X-ray machines or dental radiography systems that emit X-rays for diagnostic imaging purposes.

The film holder consists of two elements: the positioning rod in stainless steel and the bite block in plastic. The bite block is fastened to the positioning rod. The two elements need to be used together as the gap between the positioning rod and the

bite block is where the x-ray film is positioned.

X-ray images of teeth are used to establish diagnosis and to monitor a variety of conditions and their treatment. Eggen-bite facilitates this procedure. The dental X-ray film holder is not intended for any other use apart from dental radiography. It should not be used for medical or diagnostic procedures outside the field of dentistry. Additionally, it is not designed to diagnose or treat dental conditions but rather to aid in the acquisition of accurate dental X-ray images for subsequent diagnosis and treatment planning by qualified dental professionals.

The product is not delivered sterile from the manufacturer. The positioning rod is sterilized by the dentist before first use and after each use by the method of autoclaving. The bite block is discarded after use.

The film holder is delivered in three variants; straight, left and right.

**Intended Use**

The Dental X-ray Film Holder is a medical device intended for use in holding and positioning dental X-ray films during intraoral radiographic procedures. This device assists in stabilizing the film in the patient's mouth to capture consistent and clear radiographic images while minimizing movement, thus ensuring patient comfort and reducing the risk of image distortion.

**Components**

The Eggen x-Ray Holder consists of the following components:

Film holder/positioning arm: A stainless steel component designed to securely hold x-ray films and allow for precise positioning. There are three versions:

A. Straight, for anterior radiographs.

B. Right, for posterior radiographs in the upper right quadrant and lower left quadrant.

C. Left, for posterior radiographs in the upper left quadrant and lower right quadrant.

Bite block: A comfortable and disposable block for patients to bite on, ensuring stability during the radiograph and to fasten the x-ray film.

**Precautions:**

This product is for use by trained dental professionals only.

Carefully inspect the dental X-ray film holder before each use. Do not use if there are signs of corrosion, pitting, cracks, wear and tear, bent or distorted shape, sharp edges, discoloration or damage.

Follow all local, regional, and national regulations and guidelines related to X-ray imaging.

Keep the dental X-ray film holder away from excessive heat, moisture, and direct sunlight.

Do not modify or alter the dental X-ray film holder in any way.

Keep the product out of the reach of children.

The bite block is designed for single use only and cannot be reused.

**Usage Instructions:**

Ensure that the X-ray machine and all necessary equipment are properly functioning and calibrated.

Select the appropriate X-ray film size and suitable model of Dental X-ray film holder for the specific dental examination.

Wash and disinfect your hands and use gloves before handling the X-ray film holder.

Place the X-ray film inside the designated slot on the X-ray film holder. Ensure the film is positioned correctly and centered within the holder's window and securely fastened to avoid distortion.

Instruct the patient to open their mouth and gently insert the bite block, aligning the film with the area of interest.

Position the dental X-ray film holder in the patient's mouth as directed by the dentist or radiologist.

Ask the patient to hold and bite down still during the X-ray exposure and ensuring a comfortable and stable fit.

Activate the X-ray machine to capture the image. Follow the manufacturer's recommended exposure settings.

After exposure, carefully remove the X-ray film holder from the patient's mouth.

Safely remove bite block from metal holder.

### Cleaning and Maintenance:

Ensure that the metal dental X-ray film holder is free of debris, leftover films, or any other contaminants. Clean it thoroughly using a mild detergent and water.

Rinse the holder with distilled water to remove any detergent residue.

Place the cleaned and rinsed metal dental X-ray film holder inside an autoclave-safe sterilization pouch. Make sure the pouch is appropriately sized to accommodate the holder and that it's not overfilled.

If you have indicator tape or strips, place a piece on the pouch. These indicators change color when exposed to the proper sterilization conditions, indicating successful sterilization.

Check the autoclave's user manual for specific sterilization settings for metal instruments. Generally, use a standard cycle for metal instruments, which includes exposure to steam at high pressure and temperature.

Fill the autoclave's reservoir with distilled water following the manufacturer's instructions.

Check the indicator tape or strip to ensure it has changed color, indicating successful sterilization.

Inspect the metal dental X-ray film holder for any signs of damage or discoloration. If it appears damaged, do not use it and replace it with a new one.

Store the sterilized holder in a clean and dry environment, preferably in a covered container or pouch to maintain its sterility until it's ready for use.

### Troubleshooting:

Problem: Image quality is compromised.

Solution: Ensure that the X-ray film is properly seated within the holder and that the patient remains still during exposure. Verify that the X-ray machine is calibrated correctly.

### Reporting Serious Incidents:

If you experience any significant issues or problems with this medical device that may lead to harm or safety concerns, please report them to both the manufacturer and competent authority in your country. Your feedback is essential for ensuring device safety and effectiveness.

### Contact Information:

For product inquiries, support, or assistance, please contact:

Customer Support: [post@eggendental.no](mailto:post@eggendental.no)  
 Website: [www.eggendental.no](http://www.eggendental.no)  
 Office: Ålmoveien 1b, 0282 Oslo, Norway

Please retain this IFU for future reference.

### Disposal:

Dispose of the dental X-ray film holder in accordance with local regulations for medical waste disposal. Do not dispose of it in regular waste bins.

### Description of Label

An explanation of what each of the symbols used on the label means is given below:

-  REF Indicates device identification number in manufacturer system
-  LOT Indicates manufacturer's LOT NUMBER
-  Indicates the month when the medical device was manufactured
-  Indicates that this medical device has not been subjected to a sterilization process
-  CE Indicates the medical device manufacturer's claim that a product meets the General Safety and Performance Requirements (GSPR) of all relevant European Medical Device Regulations and is a legal requirement to place a device on the market in the European Union
-  MD Device is a medical device
-  Indicates caution is necessary when operating the device
-  Indicates that the device must not be used if the package holding the device is damaged
-  Indicates the need for the user to consult the instruction for use
-  Indicate medical device manufacturer, registered trademark of the manufacturer and the address
-  UDI Indicates a carrier that contains unique device identifier information
-  10% - 85% Indicates the range of humidity to which the medical device can be safely exposed
-  5°C - 50°C Indicates the temperature limits to which the medical device can be safely exposed
-  Indicates a medical device that is intended for one single use only
-  Indicate the date after which the medical device is not to be used

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