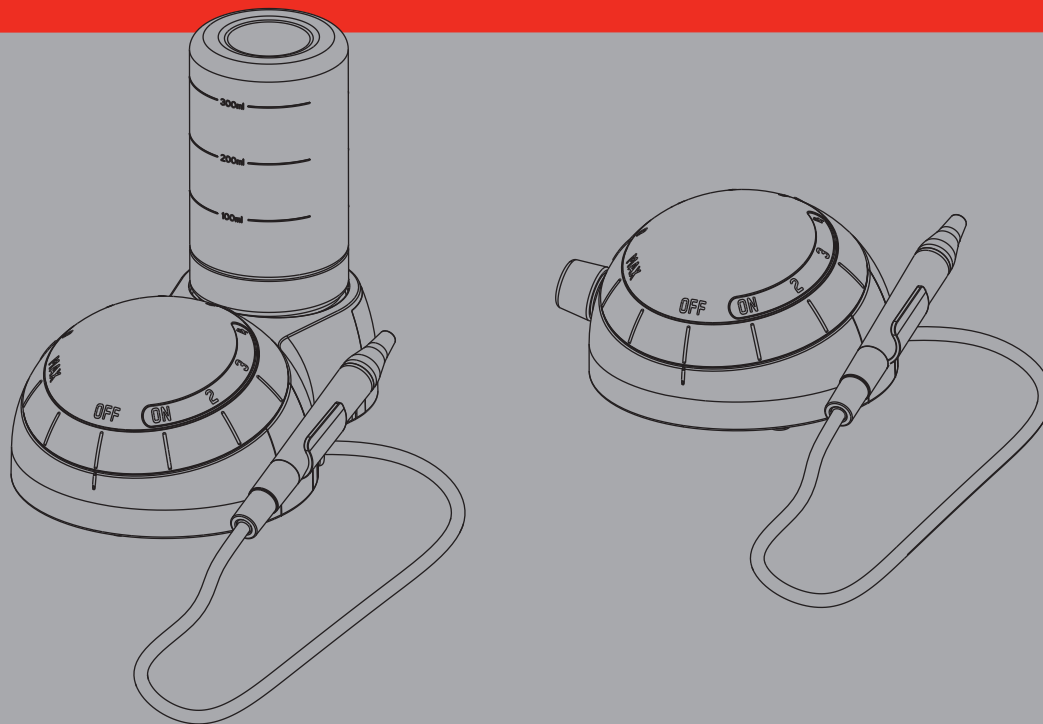
Rx ONLY Caution! Federal (USA) law restricts this device to sale by or on the order of a dentist.



OPERATION INSTRUCTIONS
UPUTE ZA RUKOVANJE
GEBRUIKSINSTRUCTIES
INSTRUÇÕES DE UTILIZAÇÃO
DRIFTSVEJLEDNING
BRUKSANVISNING

Piezon® 150
Piezon® 250


EMS+

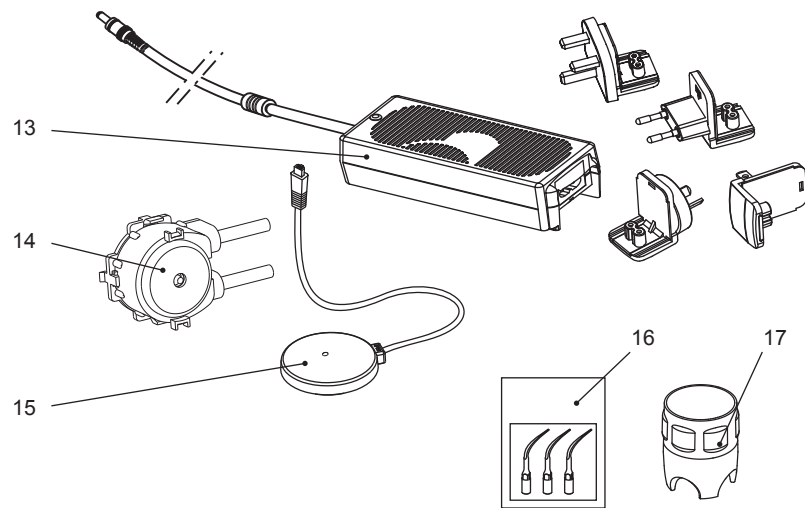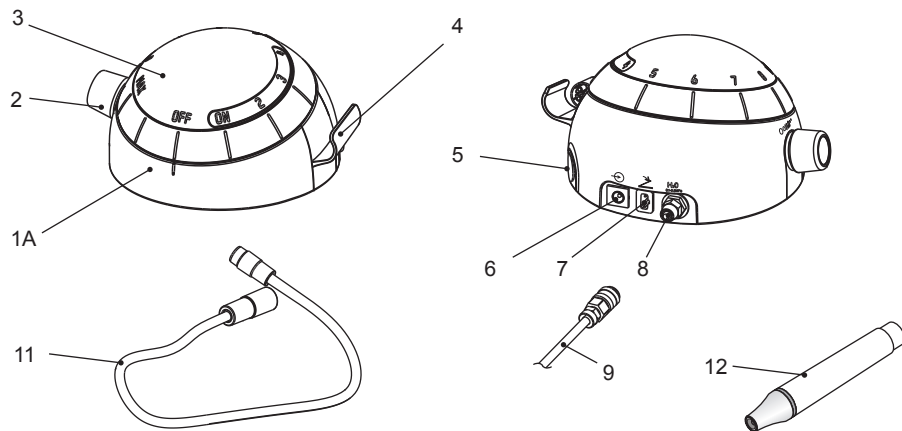Piezon 150

Piezon 250

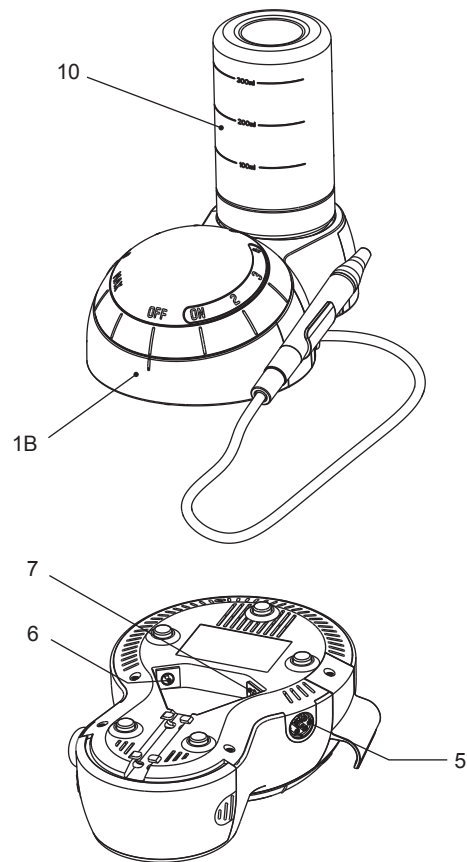COMPONENTS

- Unit
 - Piezon 150
 - Piezon 250
- Water flow rate adjustment
- Power knob
- Handpiece holder
- Handpiece connection
- External power supply connection
- Foot pedal connection
- Water hose connection
- Water supply hose
- Bottle
- Handpiece cord
- Handpiece (applied part)
- Power supply
- Pump
- Foot pedal (2 steps)
- Scaling tips (applied part)
- CombiTorque

EMS supply units with various accessories. The "Packing list" shows exactly what is included with your unit.

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DEAR CUSTOMERS,

Thank you for buying a new EMS product. This device meets the highest quality and safety standards.

In order to get the best out of this high-end dental product and ensure the longest working life possible, please read this manual carefully before using or maintaining the product.

Qualified use

This product is only intended for qualified use in dental medicine according to the applicable health and safety regulations, accident prevention measures and in observance of these instructions for use. The product must only be prepared and maintained by persons who have been instructed in infection protection, personal protection and patient safety. Improper use (e.g. due to lack of hygiene or maintenance), non-compliance with our instructions or the use of accessories and spare parts not approved by EMS invalidates all claims under warranty and any other claims.

Qualifications of the user

The product has been developed and designed for the target group "dentists, dental hygienists, dental employees (prophylaxis) and dental assistants".

For further details on the various instruments and their scopes of application, please refer to the operation instructions supplied with your Piezon system or available from the distributor in your country.

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ABOUT THIS MANUAL

 The illustrations in these instructions represent the Piezon 150 (unless otherwise specified); however, these instructions also apply to the Piezon 250.

These operating instructions are to ensure the correct installation and use of this product. Always keep this document close at hand.

Please read these operating instructions carefully as they explain the most important details and procedures. Please pay special attention to the safety precautions.

Misuse can damage the product and hence cause risks and hazards for patients, users and third parties.

Instructions concerning the components of the system are provided separately in the relative instructions for use. Read the instruction manuals originally delivered with each of the components before using this product. Do not hesitate to contact your local EMS dealer for further information.

This product needs special precautions regarding electromagnetic compatibility and needs to be installed and put into service according to the "Electromagnetic compatibility" manual.

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To prevent possible patient or user injury or damage to the Piezon150 and 250, please note the corresponding directives:

Danger

Risk of patient or user injury

Caution

Risk of damage to the device or environmental harm

Please note

Useful additional information and hints

Prohibited

Authorized

Note that the English version of this manual is the master from which the translations derive. In case of any discrepancy, the binding version is the English text.

We would be pleased to answer your questions or listen to your suggestions. We do, of course, provide support in case of technical problems. Please call your approved EMS repair center or your dealer directly.

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EMS

CONTENTS

Treatment	2
Indications	2
Contraindication	2
Installation and connection	4
Installing the pump (Piezon 250)	4
Connecting the unit	6
Electrical safety precautions	10
Preparing for treatment	12
Mode and instruments	12
Connecting the handpiece	14
Attaching the instruments	16
Installing the bottle (Piezon 250)	18
Solutions compatible with the circuit	20
Operating the unit	22
Setting the ultrasound power	24
"Perio" mode	26
"Scaling" mode	28
Setting the liquid flow rate	30
Piezon handpiece LED (Option)	32
Foot pedal functions	34
End of treatment	36
Circuit washing	38
Handpiece disassembly	40
Cleaning, disinfecting and sterilizing	42
Cleaning, disinfecting and sterilizing the handpiece and the instrument	42
Cleaning the bottle and the unit	44
Disinfecting the circuit	46
Maintenance	48
Verification of O-rings	50
Replacing the pump (Piezon 250) and the filter (Piezon 150)	52
Safety precautions	54
Storing the product	58
Disposal	60
Warranty	60
Accessories	60
EMS Service	62
Symbols	64
Technical data	70
Troubleshooting	74

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TREATMENT

Indications

This product is designed for use in dentistry with EMS ultrasound instruments for the following indications:

Scaling

- Removal of supragingival calculus
- Removal of stains

Endo

- Preparation, cleaning and irrigation of root canals
- Retrograde preparation of root canals
- Condensing gutta-percha
- Removal of crowns, bridges and restorations

Restorative

- Preparation of cavities
- Cementation of restorations
- Condensing of amalgams

Periodontics

- Scaling and root planing
- Periodontal therapy

Contraindication

 Ultrasonic oscillations may prevent cardiac pacemakers and defibrillators from functioning properly. Therefore, we recommend that patients with a cardiac pacemaker or a defibrillator should not be treated with this product.

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
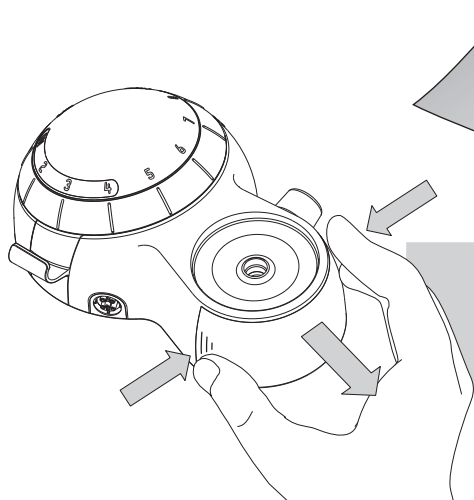
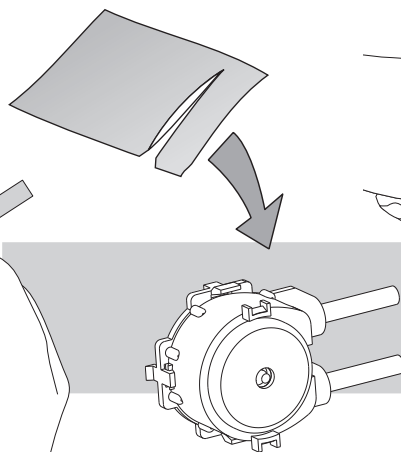
Instaliranje pumpe (Piezon 250)

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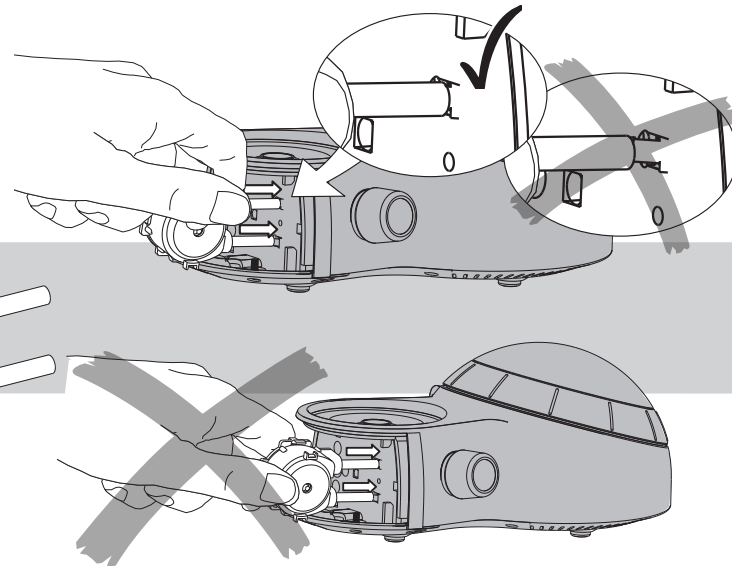Installeren van de pomp (Piezon 250)

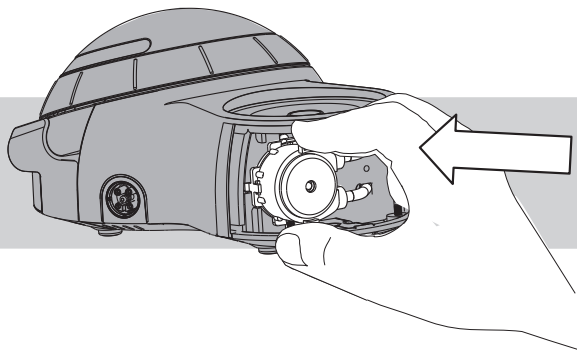
 Ensure that the pump head cover is correctly closed, before using the device.

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
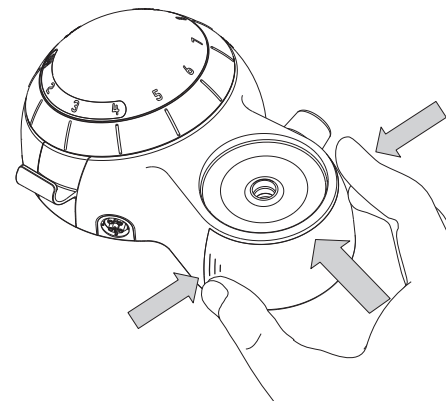
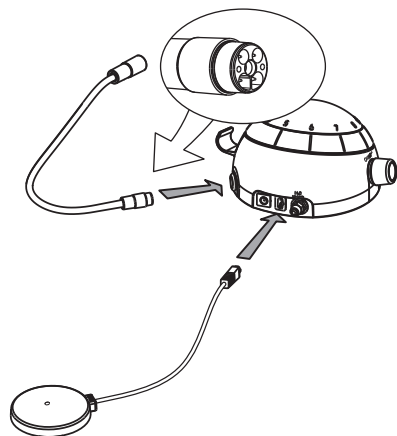
Connecting the unit

Priključivanje jedinice

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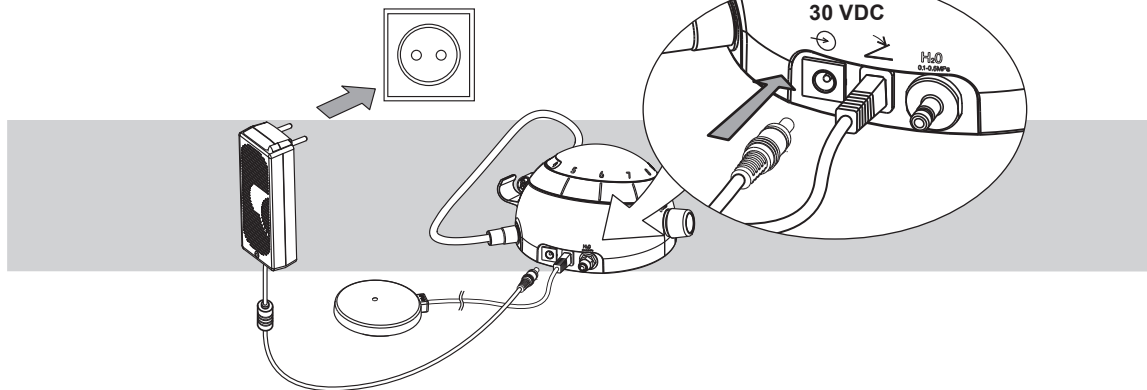 Always check that the unit is installed in a safe and stable position. A possible fall may suddenly pull on the cord and the handpiece, hurting the patient and the user, and damaging the unit.

 Check that the rated voltage of the power supply corresponds to the local line voltage to prevent damaging the unit.

Use the unit only in a FI protected mains supply (FI = Residual current protection).

 For USA or Canada: connect only to a Hospital Grade outlet.

Connect the unit to a water supply with the original EMS water hose supplied with your product.

 It is recommended to use an independent water supply.

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
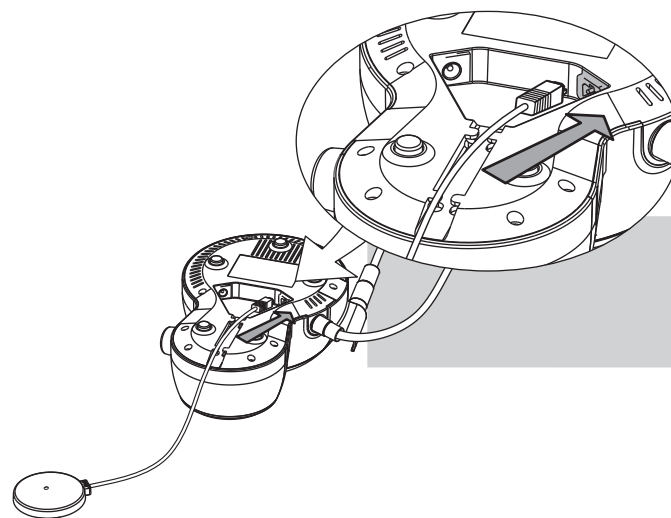
To connect the foot pedal and the power supply to the Piezon 250, turn over the unit as shown in the pictures above.

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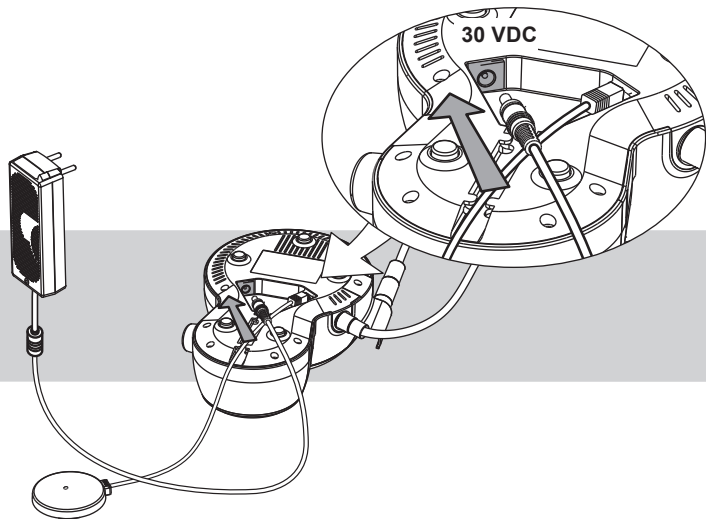Electrical safety precautions

 No modification of the unit and the accessories supplied with the unit are allowed.

 Do not touch the connector of the handpiece cord and the patient simultaneously.

 The power supply must not be fixed tightly by screw.

The power supply can be placed on a table or hung on the wall. So it is accessible and can easily be detached.

 To disconnect the unit from the supply network, unplug the power supply.

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Mode and instruments

Scaling

 Avoid using the HVE (high volume evacuation) to ensure sufficient cooling of the tooth and the instrument. The instrument heats up quickly if the irrigation flow rate is not sufficient.

 Instruments: please refer to the "Piezon Systems" manual.

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
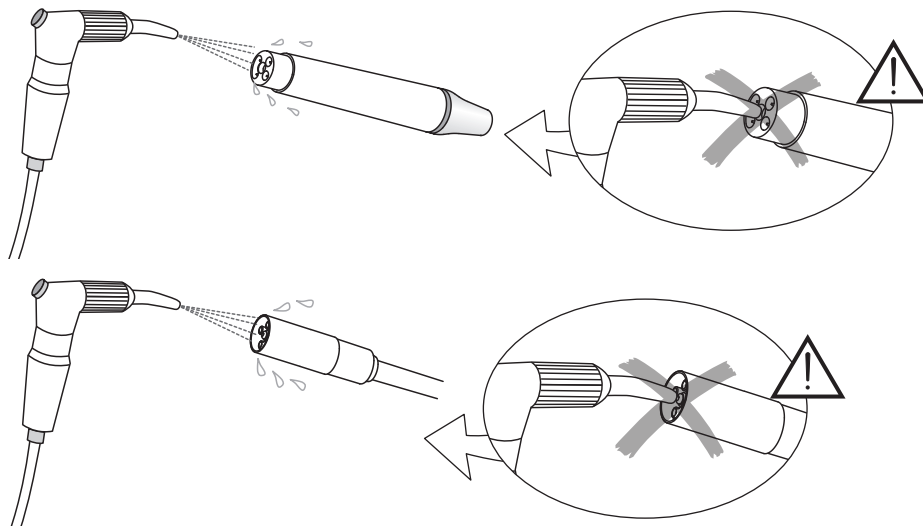
Connecting the handpiece

Priključivanje ručne jedinice

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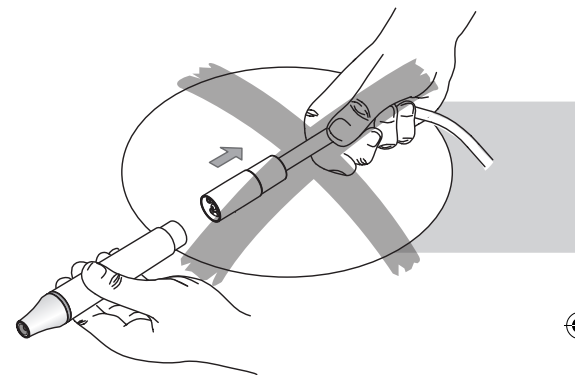 Clean, disinfect and sterilize the different pieces and accessories of this product before each use. Please refer to the "Cleaning, disinfecting and sterilizing" chapter. Non-sterile pieces and accessories may cause bacterial or viral infections.

 Blow dry the connections to remove the possible presence of liquid to ensure a proper electrical contact. Never blow compressed air in the irrigation connections as this will irretrievably damage internal parts.

 Očistite, dezinficirajte i sterilizirajte različite dijelove i dodatke uređaja prije svake upotrebe. Više informacija naći ćete u poglavlju "Čišćenje, dezinfekcija i sterilizacija". Nesterilni dijelovi i dodaci mogu prouzročiti bakterijske ili virusne infekcije.

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Monter flasken med en roterende bevegelse til den stopper.

Solutions compatible with the circuit

Please read the instructions for use of the disinfecting products.

Please refer to standards applicable in your country regarding the purchase and use of these products.

Disinfecting solutions for the treatment

- Chlorhexidine ($\leq 0.3\%$)
- Physiological water ($\text{NaCl} \leq 0.9\%$)
- Hydrogen peroxide ($\text{H}_2\text{O}_2 \leq 3\%$)
- Carbanilides
- Quaternary ammonium compounds
- Citric acid
- Aromatic essence
- Ethanol

Examples of common products

BacterX® pro / Triclosan / Listerine® / Alodont® / Meridol® / BreathRx® / CloSYS™ / Scope® / Cepacol® /.....

Disinfecting solutions for washing the circuit

- EMS Ultra Clean (H_2O_2 6%)
- Hydrogen peroxide (H_2O_2 5%)
- Sterilox (hypochlorous acid)

Incompatible mixing

Different disinfecting solutions can be used during treatment. Chemical reactions might occur between these solutions and damage the unit. Wash the liquid circuit for 20 seconds with distilled (or demineralized) water before using a different disinfecting solution (please refer to the "End of treatment" chapter).

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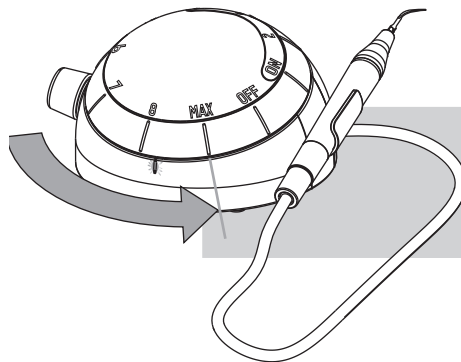
Način rada "Uklanjanje zubnog
kamenca"

"Scaling" modus

 The "Scaling" mode uses the entire power range for the ultrasounds, from "ON" to "MAX".

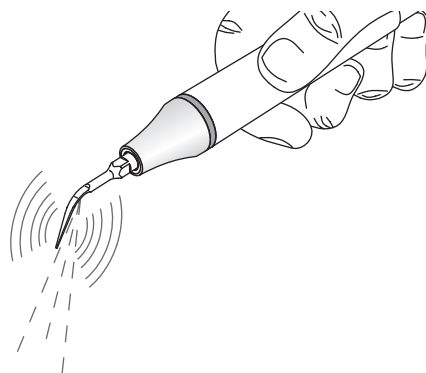
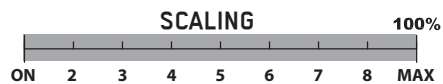
 U načinu rada "Uklanjanje zubnog kamenca" koristi se cijeli raspon snage za ultrazvuk, od "ON" do "MAX".

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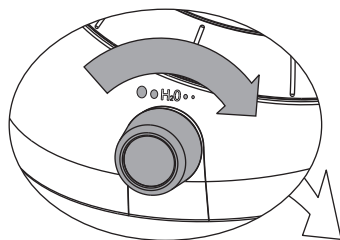
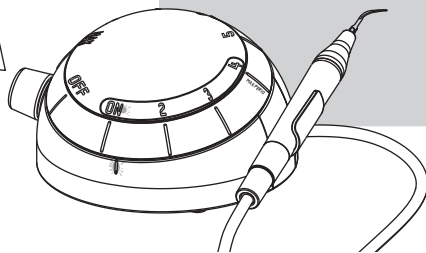
 Optimal settings for the liquid flow rate for each instrument are indicated in the operating instructions of the "Piezon Systems".

 Never work dry to avoid heat damage on the tooth, unless the instrument is intended specifically for this purpose. The tip of the instrument immediately heats up when used dry.

 For further information concerning instruments usable in "DRY WORK" mode, please refer to the "Piezon Systems" manual.

 Optimalne postavke stope protoka tekućine za svaki instrument navedene su u uputama za rukovanje "Sustavi Piezon".

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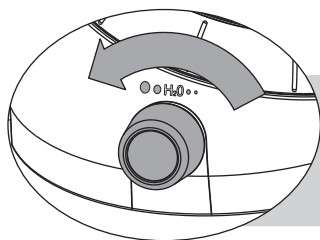
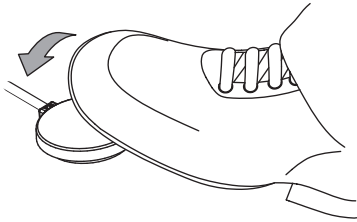
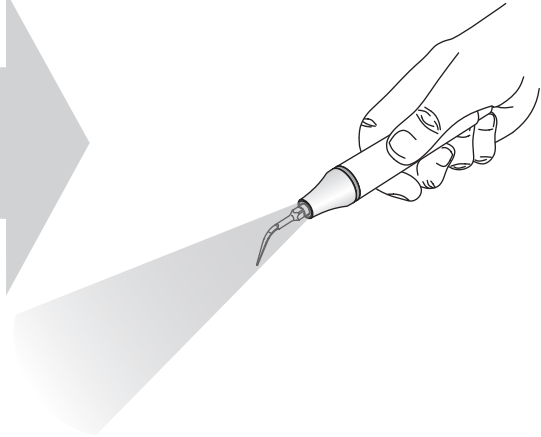
LED dioda ručne jedinice Piezon (dodatna oprema)

Piezon handstuk LED (Optie)

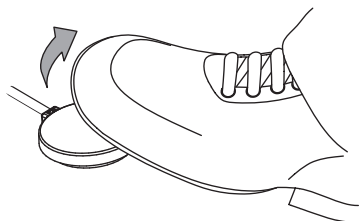
 The Piezon handpiece LED lights up when the foot pedal is activated. It remains lit for a duration of 20 seconds after the foot pedal is released.

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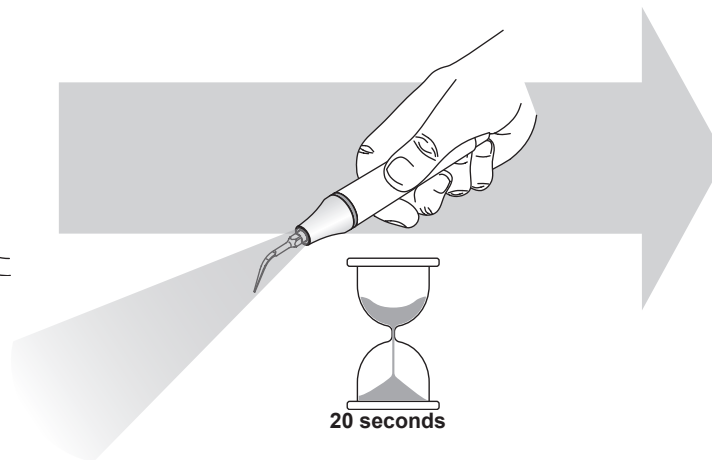
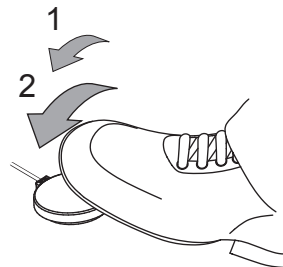
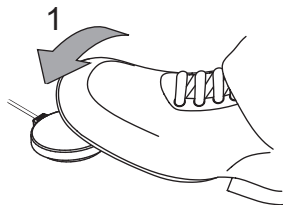
Foot pedal functions

Ultrasounds with liquid

 Upon first use, keep pressure on the pedal until the water goes out off the handpiece.

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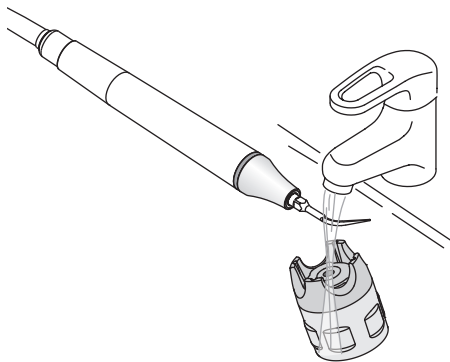
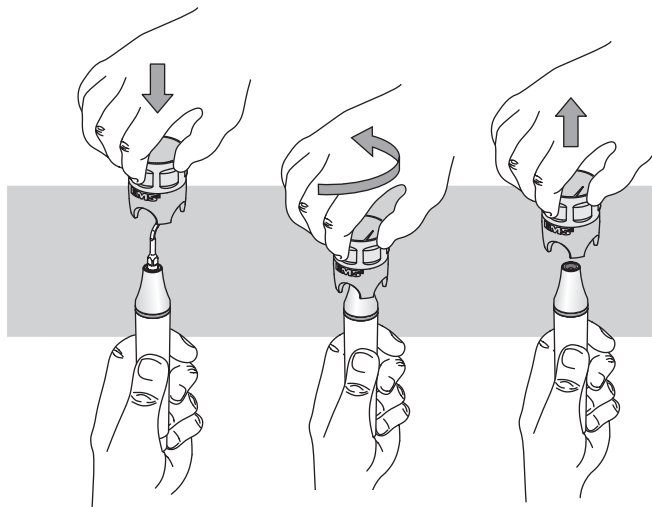
END OF TREATMENT

ZAVRŠETAK TRETMANA

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 Clean the instrument and the CombiTorque with running water before disinfection.

 Clean the unit and the accessories (please refer to the "Cleaning, disinfecting and sterilizing" chapter).

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
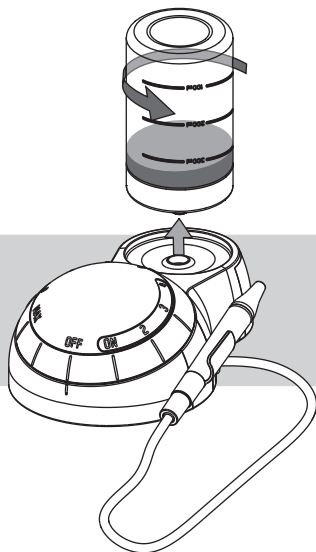
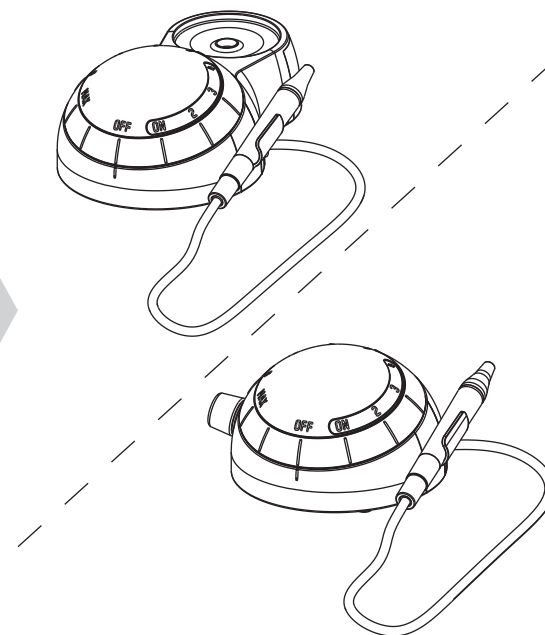
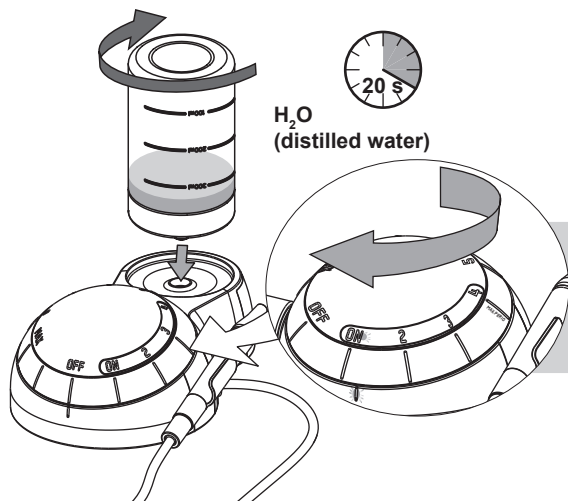
Circuit washing

Pranje kruga

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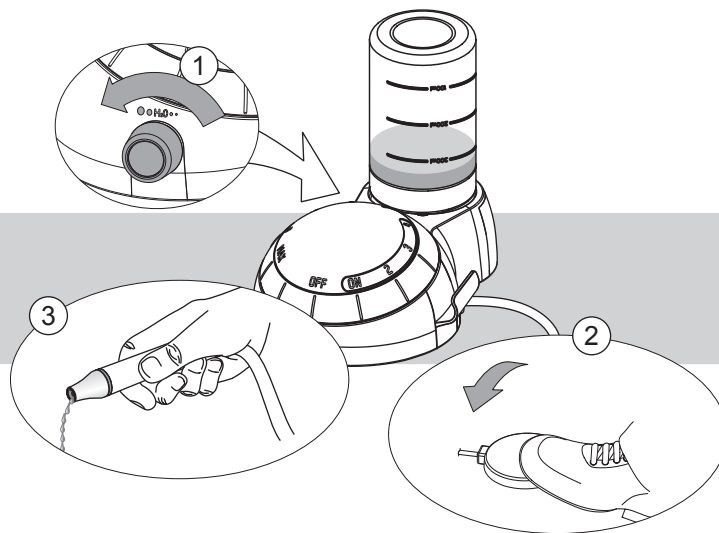⚠ Once the treatment has been completed, wash the circuit during 20 sec. of distilled (or demineralized) water to avoid a possible reaction caused by ageing of the solution or by an inadvertent mixture of liquids.

👉 Clean the unit and the accessories (please refer to the "Cleaning, disinfecting and sterilizing" chapter).

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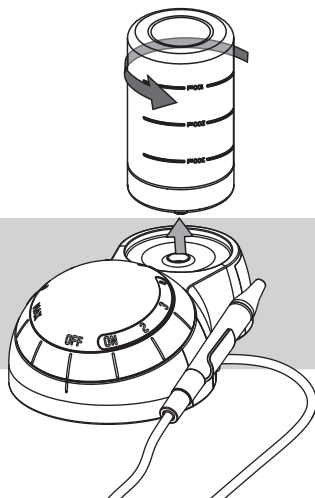
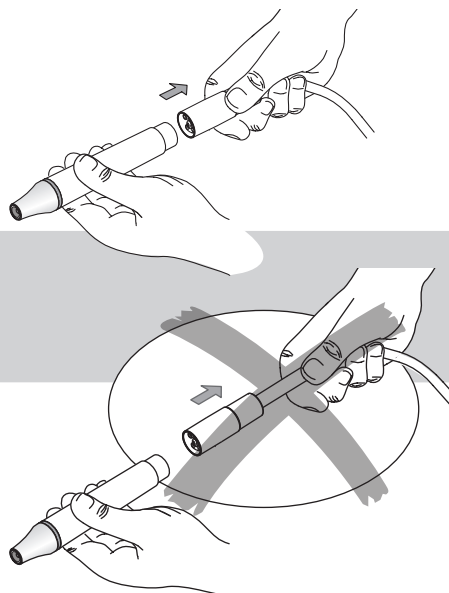
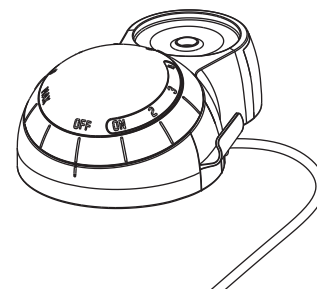
Handpiece disassembly

Rastavljanje ručne jedinice

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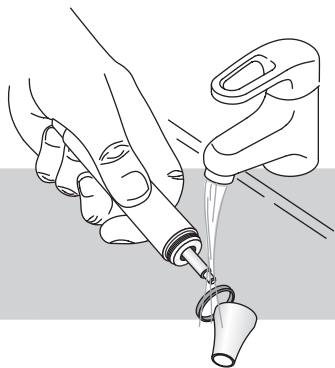
 Unscrew the cap and the light guide (when equipped) under running water before cleaning.

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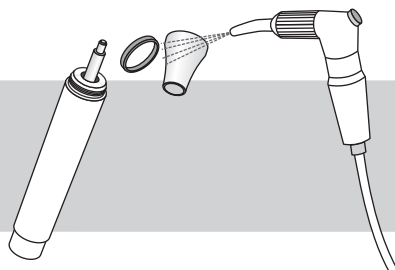
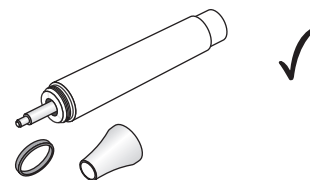
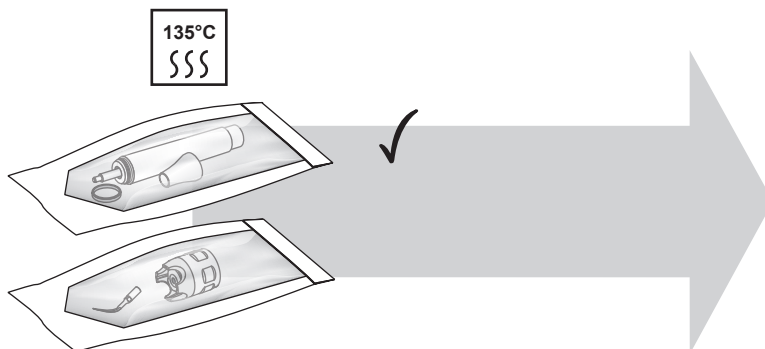
CLEANING, DISINFECTING AND STERILIZING

Cleaning, disinfecting and sterilizing the handpiece and the instrument

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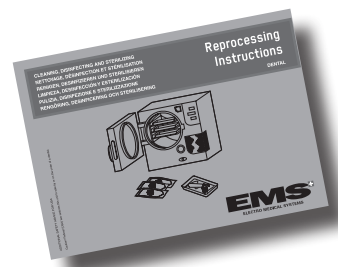
 Please follow the recommendations of the reprocessing instructions manual delivered with your product regarding the procedures for cleaning, disinfecting, sterilizing and packing of the components (handpiece and instruments).

 Slijedite preporuke u priručniku s uputama za ponovnu obradu isporučenom s vašim proizvodom u vezi s postupcima čišćenja, dezinfekcije, sterilizacije i pakiranja komponenata (držka i instrumenti).

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
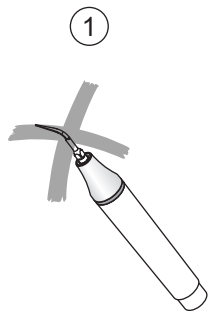
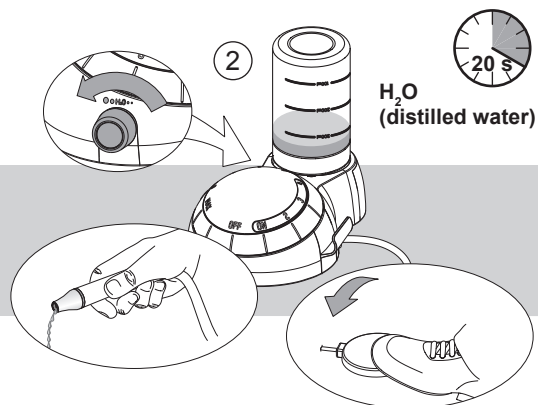
Disinfecting the circuit

Dezinfekcija kruga

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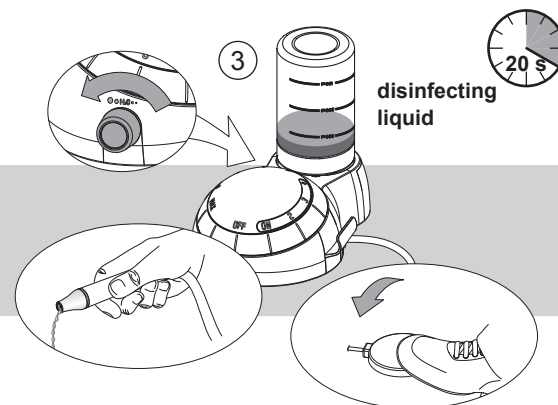 It is recommended to disinfect the liquid circuit daily.

 Disinfection is necessary to reduce crystalline deposits and the number of bacteria in the liquid circuit of the unit; the following procedure achieves a 5 log reduction of target pathogens (using disinfecting solutions for washing the circuit as listed at the beginning of this manual):

- 1 Remove the instrument,
- 2 Wash the circuit during 20 seconds with distilled (or demineralized) water,
- 3 Disinfect the circuit during 20 seconds with an appropriate disinfecting solution for washing the circuit
- 4 Allow the disinfecting liquid to act for 5 minutes,
- 5 Rinse the circuit during 20 seconds with sterile water.

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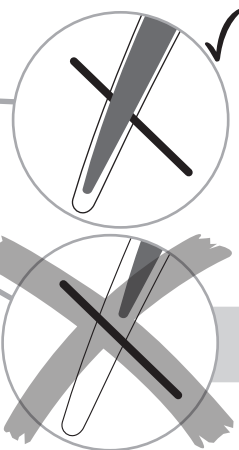
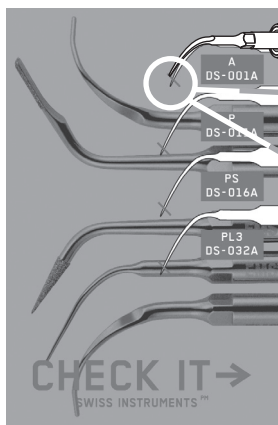
ODRŽAVANJE

ONDERHOUD

The EMS Piezon instruments must be checked regularly using the EMS "Tip Card". Ultrasound instruments wear during use and become shorter. Worn instruments are ineffective and are a cause of discomfort to the patient.

-  Instruments showing wear on the diamond coating must be replaced.
-  For precautionary reasons, do not exceed the validated lifespan of the components (see the Technical Data chapter).
-  Always use EMS original parts. Using non-original components may damage the equipment, and practitioner or patient may be injured.

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SAFETY PRECAUTIONS

EMS and the dealer of this product accept no liability for direct or consequential injury or damage resulting from improper use, arising in particular through non-observance of the operating instructions, or improper preparation and maintenance.

 Use for the intended purpose only: before using the product, make sure that you have studied the operating instructions. This also applies to any equipment used with this product. Failure to observe the operating instructions may result in the patient or user suffering serious injury or the product being damaged, possibly beyond repair.

 Risk of explosion: do not use this product in the presence of inflammable anaesthetics or gases.

 The operator shall not touch the cord connector and the patient simultaneously.

 During treatment, avoid touching the patient with the handpiece body, as the heat from the handpiece can be uncomfortable for the patient.

 No specific training other than initial professional training is required to use this medical device. The practitioner is responsible for performing the clinical treatments and for any dangers that may arise due to a lack of skill and/or training.

 Before each treatment, check the contents of the bottle as well as the liquid present in the circuit.

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 Clean, disinfect and sterilize the different pieces and accessories of this product before each use. Please refer to the information provided in this operating instructions. Non-sterile pieces and accessories may cause bacterial or viral infections.

 This product has been investigated with regard to safety from electrical shock and fire hazard. ETL (Electrical Testing Laboratories) has not investigated the physiological effects.

 Always examine the product for damage before commencing treatment. Damaged accessories or a damaged unit must not be used and must be replaced. Use original EMS spare parts and accessories only.

 Have this product exclusively repaired by an approved EMS repair center.

 Use exclusively the external power supply (ref EK-357) delivered with the EMS unit.

 Use exclusively the pedal (ref EK-128) delivered with the EMS unit.

 Certain types of mobile telecommunication equipment could potentially interfere with this product. The separation distances recommended in the "Electromagnetic compatibility" manual must be taken into account.

 Turn off the water inlet when not in use. The device is not equipped with Aquastop and the water hose may disconnect or leak: risk of flooding.

 Očistite, dezinficirajte i sterilizirajte različite dijelove i dodatke uređaja prije svake upotrebe. Više o tome pročitajte u ovim uputama za rukovanje. Nesterilni dijelovi i dodaci mogu prouzročiti bakterijske ili virusne infekcije.

 Ispitana je sigurnost ovog proizvoda na opasnost od električnog udara i požara. Laboratoriji za električna ispitivanja (ETL – electrical testing laboratories) nisu ispitali fiziološke utjecaje.

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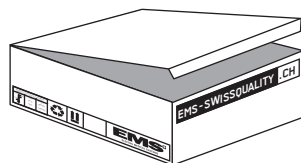
STORING THE PRODUCT

SKLADIŠTENJE PROIZVODA

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Keep the original packaging until the product is to be disposed of permanently. It can be used for shipping or storing at any time.

Should you wish to put your product out of use for an extended period of time:

- Proceed as described in the "Cleaning, disinfecting and sterilizing" chapter,
- Proceed to a washing of the circuit without bottle to empty it,
- Pack the product and all accessories in the original packaging.

Please refer to the storage and transport conditions in the "Technical data" chapter.

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DISPOSAL

The product must not be discarded in domestic household waste.

Should you wish to definitively dispose of the product, please comply with the regulations which apply in your country.

Waste Electrical and Electronic Equipment belonging to customers located in the European Union may be shipped to EMS for recycling in accordance to the WEEE regulations. The costs of recycling, exclusive of shipping fees, are covered by EMS.

WARRANTY

Damage due to non-adherence to the operating instructions or resulting from the wear out of parts are excluded from warranty.

 The warranty of your product will be cancelled if you try to open it.

ACCESSORIES

Accessories are available from EMS or any authorized dealers. Please contact your customer service directly.

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EMS SERVICE

Should your product need additional servicing or repairs, please send it to your dealer or to an authorized EMS Repair Center.

In the case of non-authorized repairs or damages due to non-adherence to the operating instructions, EMS accepts no liability whatsoever. This will also void the warranty.

It is best to ship your product in the original packaging. It protects your product against damage during shipment.

 Risk of transmitting micro-organisms! To protect the personnel of the repair center and for safety reasons during transport and shipment, the product and the accessories returned to the factory for repair or revision must be cleaned, disinfected and sterilized in accordance with the operating manual. Repair can be refused for product or accessories received in a contaminated condition.

When sending your product directly to the approved EMS repair center, please include the name of your dealer. This simplifies processing.

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TECHNICAL DATA

Description

Manufacturer	EMS ELECTRO MEDICAL SYSTEMS SA, CH-1260 Nyon, Switzerland
Models	Piezon 150 & Piezon 250
Classification EN 60601-1	- Class II - Piezon 150: Applied Part Type B - Piezon 250: Applied Part Type BF - IP 20, unit - IP X1, foot pedal
Classification 93 / 42 EEC	Class IIa
Operating mode: With irrigation Without irrigation	Continuous operation Duty cycle 10% during 10 min. maximum
Power Supply: External voltage unit	100-240 VAC / 50-60 Hz 30 VDC
Power consumption	30 W
Ultrasonic specifications: Maximum output Frequency range	8 Watts 24-32 kHz
Weight	Piezon 150: Approx. 0.6 kg Piezon 250: Approx. 1 kg
Dimensions in mm (H x W x D)	Piezon150: 96 x 170 x 135 Piezon250: 96 x 153 x 215
Operating conditions	+10 °C to +40 °C 30% to 75% relative humidity Altitude max. 3000 m
Storage and transport conditions	-10 °C to +40 °C 10% to 95% relative humidity 500 hPa to 1060 hPa air pressure

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TROUBLESHOOTING

Symptom	Solution
The LEDs of the power supply do not illuminate when the unit is switched on	<ul style="list-style-type: none">• Check that the mains cord of the power supply is properly connected to the mains supply• Check the power supply of the dental office• Turn the power knob of the unit to ON and check if the LED is lit• Replace the transformer• Send the unit and the accessories to an approved EMS repair center
No irrigation flow when the foot pedal is pressed	<ul style="list-style-type: none">• Turn the liquid flow rate knob to the maximum• Make sure that water comes in external device for the Piezon 150• Make sure that the bottle is properly installed for the Piezon 250• Disconnect the ultrasound handpiece and check that the liquid exits from the cord• Use compressed air (max. 3 bar) and check that air comes out of the handpiece• If the air does not come out of the handpiece send to an approved EMS repair center• Replace the water filter for the Piezon 150• Replace the pump head for the Piezon 250• Check the connection of the foot pedal• Replace the foot pedal• Send the unit and the accessories to an approved EMS repair center
No ultrasound oscillations when the foot pedal is pressed	<ul style="list-style-type: none">• Check that the handpiece is connected to the handpiece cord• Control that the instrument is tight• Check that the instrument is not cracked• Check the connection of the foot pedal• Replace the foot pedal• Send the unit and the accessories to an approved EMS repair center
Ultrasound power decreasing or inadequate	<ul style="list-style-type: none">• Check that the selector is beyond the position 4• Control that the instrument is tight• Check the wear of the instrument and replace if necessary• Replace the handpiece• Send the unit and the accessories to an approved EMS repair center
Water leaks between the handpiece cord and the handpiece	<ul style="list-style-type: none">• Replace the O-ring on the handpiece cord• Replace the handpiece• Send the handpiece to an approved EMS repair center
<i>Only Piezon 250:</i> Water leaks from the bottle or the device	<ul style="list-style-type: none">• Check the O-ring of the bottle cap• Replace the bottle• Send the handpiece to an approved EMS repair center
Insufficient lighting of the handpiece	<ul style="list-style-type: none">• Replace the light guide

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EMS Italia S.r.l
EMS Corporation