

Contact form  
 By fax: +49 (0) 2451 971-410  
 By phone: +49 (0) 2451 971-409  
 By UK phone: 0800 011 2738  
 E-Mail: info@garrisondental.net

**NEW! Order now via our webshop:**  
[www.garrisondental.com](http://www.garrisondental.com)



I want to register my kit for the 60-day money-back guarantee.

**Order:** (will be supplied by your dealer)

Quantity	Part No.

Please send me your Garrison Newsletter

per post

per e-mail

Garrison Dental Solutions • Carlstrasse 50 • D-52531 Uebach-Palenberg • Germany  
 Phone: +49(0)2451 971-409 • Fax +49 (0)2451 971-410 • E-Mail: info@garrisondental.net • www.garrisondental.com

Practice \_\_\_\_\_

No. Street \_\_\_\_\_

Town/City \_\_\_\_\_

Postcode \_\_\_\_\_

Phone \_\_\_\_\_

E-Mail \_\_\_\_\_

Dental Depot \_\_\_\_\_

in \_\_\_\_\_

Date/Signature \_\_\_\_\_

CATUK2018

**Garrison**  
 Dental Solutions

**Garrison**  
 Dental Solutions

Garrison Dental Solutions  
 Carlstrasse 50  
 52531 Uebach-Palenberg  
 Germany  
 Phone +49 (0)2451 971-409  
 Fax +49 (0)2451 971-410  
 E-Mail: info@garrisondental.net  
 Web: www.garrisondental.com

CATUK2018

**Composi-Tight 3DFusion™**  
 Sectional Matrix System


**NEW!**

When design and  
 function come  
 together...you get  
**FUSION!**



**Exclusive**  
 Wide Prep Ring

2018 product catalog  
 English

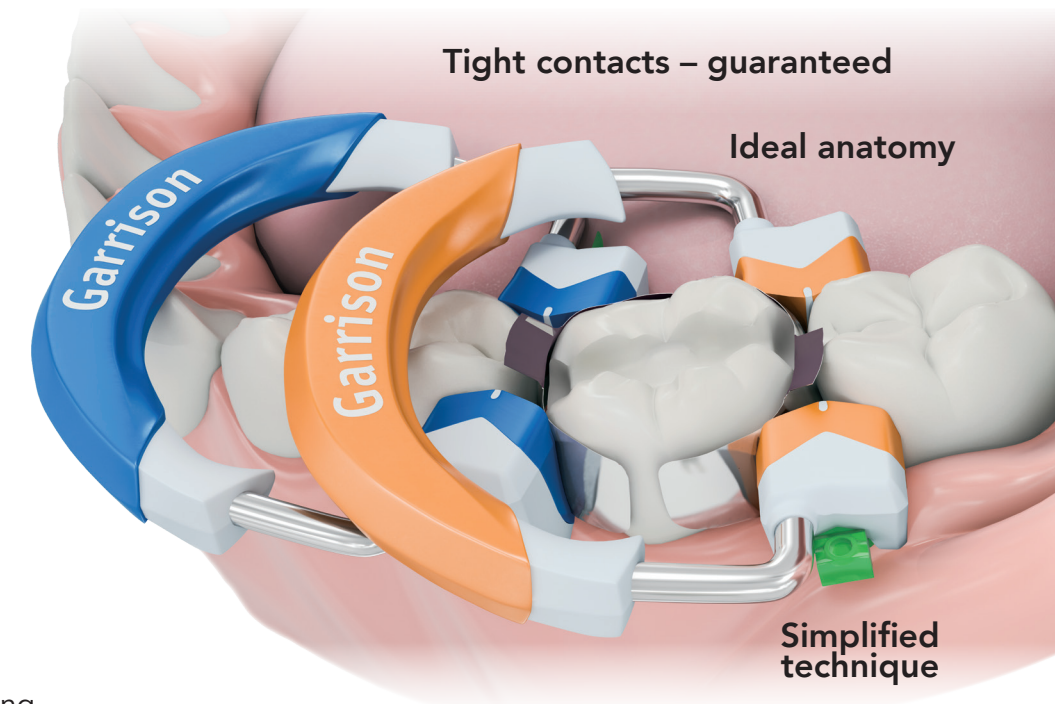

**Tight contacts – guaranteed**

**Ideal anatomy**

**Simplified  
 technique**

**Garrison**  
 Dental Solutions



**Better, faster, more efficient**  
Dr. Martin von Sontagh, Hard, Austria

ZWP Zahnarzt Wirtschaft Praxis 7+8/14, S. 72-73, Oemus Media AG, Germany



1. Pre-operative situation showing carious lesions and old fillings in tooth 14 to 17.  
2. Placement of a rubber dam, after the removal of the carious tissue.  
3. Agitation of Adhese Universal with the help of the VivaPen™ after placing of the sectional matrix system.  
4. Polymerization of the bonding agent for 20 seconds with Bluephase Style™.  
5. The filling is replaced with only two increments of tetric EvoCeram Bulk Fill™.



6. Etching of the enamel for 30 seconds.  
7. Placement of the filling with two increments.  
8. The enamel tissue is etched with phosphoric acid for 30 seconds while the dentin tissue is etched for 15 seconds.  
9. The Adhese Universal bonding agent is easy to dispense from the VivaPen™.  
10. The completed fillings smoothly blend into the dental arch, due to their natural-looking cusps and fissures.

You can find the complete case for download at [www.garrisondental.com](http://www.garrisondental.com).

© by Ivoclar Vivadent AG

Tight, predictable flash-free contacts

**Composi-Tight® 3DFusion™**  
Sectional Matrix System



Excellent clinical results with PROVEN materials

**Why use a sectional matrix system for Class II's?**

**Composi-Tight® 3DFusion™**

- Anatomically correct contacts
- Contacts at the height of contour
- Rings produce optimal tooth separation for excellent, tight contacts
- Naturally contoured bands
- Great results with an operator-friendly retaining system



**versus**

**Tofflemire-style System**

- Fails to restore proximal anatomy
- Thin contact at the marginal ridge
- Large food trap below
- Increased likelihood of fracture, occlusal interference, recurrent caries and periodontal disease



**Better contacts, virtually ZERO flash with our Composi-Tight 3DXR system!**

**General Safety Notes**

Warnings:

- Composi-Tight® systems consists of small, sharp parts. Placement of rubber dam or other suitable isolation is advised prior to operative procedures. As with all intraoral instruments, care must be exercised to ensure stable handling and placement.
- Aspiration: If aspiration of matrix, wedge or retaining ring occurs, seek immediate medical attention.
- Ingestion: If accidental swallowing occurs, encourage patient to seek medical assistance to locate swallowed part. If nausea or illness develop, seek medical attention immediately.

For full sterilization instructions please go to: [www.garrisondental.com](http://www.garrisondental.com).

Garrison Dental Solutions • Carlstrasse 50 • D-52531 Uebach-Palenberg • Phone +49(0)2451 971-409 • UK phone: 0800 011 2738 • [info@garrisondental.net](mailto:info@garrisondental.net)

**Sterilisation and cleaning instructions for Composi-Tight® 3D Fusion™, 3D XR and 3D**

Sterilisation:

- Sterilise the product in the autoclave using only CE approved sterilisation pouches.
- Only sterilise in autoclave. Run at cycle of 132 – 134°C and 210kPa for 6 minutes with a drying time of 20-30 minutes; please follow the manufacturer's instructions when doing so. Never exceed temperatures of 137°C.

- DO NOT use chemical sterilisation processes.

Cleaning:

- Remove all visible residues prior to sterilisation.
- To do this, use a nylon brush or a similar soft brush. Be careful not to damage the Soft-Face™ silicone during the cleaning process.
- If visible residues are still present, please repeat the above procedure.
- DO NOT use solutions containing chemicals such as alcohol, acetone, or ketone.

# table of contents

## Composi-Tight® 3D Fusion™ NEW

Composi-Tight® 3D Fusion™ Rings <b>NEW</b>	3
Composi-Tight® 3D Fusion™ Sectional Matrix System <b>NEW</b>	4-5
Composi-Tight® 3D Fusion™ Bands	6
Composi-Tight® 3D Fusion™ Ring Placement Forceps <b>NEW</b>	7
Composi-Tight® 3D Fusion™ Wedges	8

## Sectional Matrix Systems

Composi-Tight® 3D Fusion™ <b>NEW</b>	3
Composi-Tight® 3D XR	10
Composi-Tight® 3D Clear	12
Composi-Tight Gold®	13
Composi-Tight® Original	14

## Rings

Composi-Tight® 3D Fusion™ Green, Blue, Orange <b>NEW</b>	4-5
Composi-Tight® 3D XR	9
Composi-Tight® 3D Clear	12
Composi-Tight Gold®	13
Composi-Tight® Original	14

## Bands

Composi-Tight® 3D Fusion™ <b>NEW</b>	6
Composi-Tight® Slick Bands™ XR Matrices	11
Composi-Tight® 3D Matrix Bands	11
Composi-Tight® 3D Clear Matrix Bands	12
Composi-Tight Gold® Matrix Bands	13
Composi-Tight® Original Bantrix Bands	14

## Anterior Matrix Systems

VariStrip	15
-----------	----

## Traditional

Tofflemire-style	Slick Bands™ Margin Elevation Bands	5, 19
	Slick Bands™ Tofflemire-style	19-20

## Wedges

Composi-Tight® 3D Fusion™ Wedges <b>NEW</b>	8
WedgeWands® Plastic Wedges with Applicator	16
A+Wedge® Plastic Wedges with Astringent	18
G-Wedge™ Plastic Wedges	18
SoftWedge™ Wooden Wedges	17

## Important Accessories

Matrix Forceps	Matrix Forceps	27
	Composi-Tight® 3D Fusion™ Ring Placement Forceps <b>NEW</b>	7
	Ring Placement Forceps	27
	Universal Forceps	27

## Instruments

PerForm™ Contact Former	21
Multi-Function Non-Stick 5-in-1 Instruments	28
Double Cord Packer	29
Composite Instruments	29-30

## Polishers

FitStrip™ Interproximal Finishing and Contouring System	23-24
Rally™ Mini Polishers for Composite and Hybrid Ceramics	22
eZr™ Zirconia and Lithium Disilicate Polishers	25-26

## Key Opinion Leaders

Statements of Garrison Users	31-32
------------------------------	-------

## Sterilization and cleaning instructions

35

Errors and specifications subject to change without notice.

# Composi-Tight® 3D Fusion™

## Sectional Matrix System



All sets have a 60-day money-back guarantee. Please register your set today by e-mail to [info@garrisdental.net](mailto:info@garrisdental.net), by phone on 0800 011 2738 or by sending the order form by fax to +49 (0) 2451 971-410.

### NEW!

The new Composi-Tight® 3D Fusion™ Sectional Matrix System provides the solution for all Class II cavities.

Soft-Face™ silicone tips with enhanced marginal ridge anatomy hug the matrix band to the tooth for perfect contact points and flash free restorations.

## Price: £ 449.00

Item	Description	Price
<b>FX-KFF-00 Starter Set</b>	1x Composi-Tight® 3D Fusion™ Ring Short blue, 1x Composi-Tight® 3D Fusion™ Ring Tall orange, 1x Composi-Tight® 3D Fusion™ Ring Wide Prep green, 80x Composi-Tight® 3D Fusion™ Wedges, 4 sizes, 70x Composi-Tight® 3D Fusion™ Matrix Bands, 5 sizes, 1x Ring Placement Forceps	£ 449.00
<b>FX-KFF-04</b>	Composi-Tight® 3D Fusion™ FX-KFF-00 Starter Set with 1x BFA-1 Matrix Forceps (p. 23)	£ 575.00
<b>FX-KFF-01</b>	Composi-Tight® 3D Fusion™ FX-KFF-00 Starter Set with 1x BFA-1 Matrix Forceps (p. 23) and 1x TN009 Multifunction Instrument (p. 27)	£ 675.00

# Composi-Tight® 3D Fusion™

## Sectional Matrix System



### The new Composit-Tight® 3D Fusion™ green ring for wide preps

This ring is great for wide embrasures, and holds even when a cusp is missing. The broad silicone tips prevent the matrix band from being squashed into the cavity.

**Part.nr. FX600 (2 pcs / pack)**



### The orange Composit-Tight® 3D Fusion™ ring now with retentive tips

Ultra-Grip™ retentive tips prevent slipping on all Composit-Tight® 3D Fusion™ rings. The improved form makes opening and placing the ring much easier. It also holds on large molars.

**Part.nr. FX500 (2 pcs / pack)**



### The popular blue Composit-Tight® 3D Fusion™ ring is now even easier to open

The new ring gives improved adaptation to the marginal ridge anatomy because of the soft silicone tips. It works very well on short and malpositioned teeth, and holds on top of a rubber dam clamp or between canine and first premolar.

**Part.nr. FX400 (2 pcs / pack)**

**NEW!**




Item	Description	Price
FX600	Composit-Tight® 3D Fusion™ Ring Wide Prep green 2 rings/pack	£ 185.00
FX500	Composit-Tight® 3D Fusion™ Ring Tall orange 2 rings/pack	£ 185.00
FX400	Composit-Tight® 3D Fusion™ Ring Short blue 2 rings/pack	£ 185.00
FXR01	Set of all 3 rings (1x green, 1x orange and 1x blue)	£ 280.00
FXP01	Composit-Tight® 3D Fusion™ Ring Placement Forceps	£ 80.00

# Composi-Tight® 3D Fusion™

## Sectional Matrix System

**NEW!**

The NEW Composi-Tight® 3D Fusion™ Sectional Matrix System provides the solution for all Class II restorations, even very deep or wide preps.

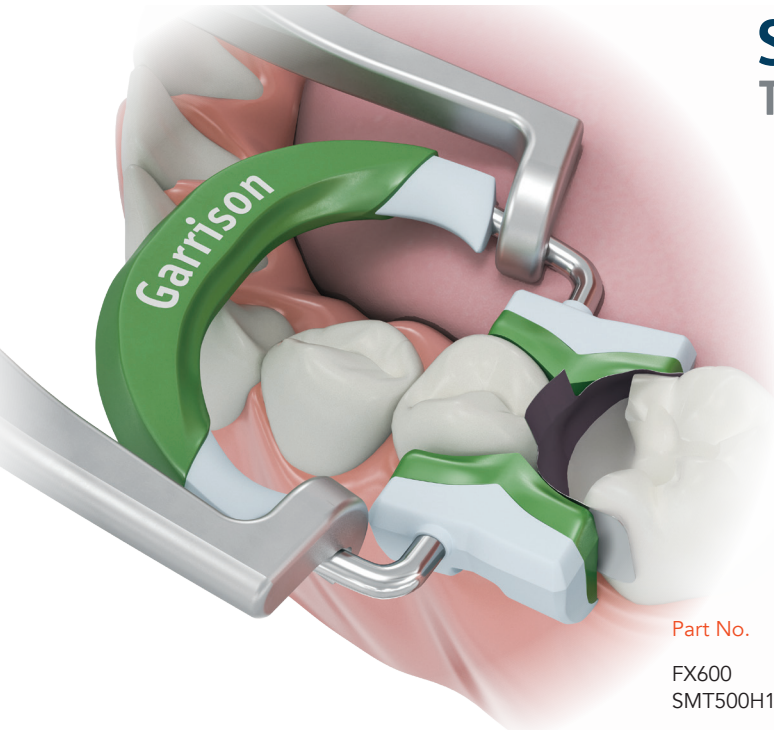

## SlickBands™



### Tofflemire-style



**Margin  
Elevation Band**

**Tip:**

- 1) Use the Margin Elevation Band (see page 19) to bring the cavity over the gingival margin.
- 2) Remove the Margin Elevation Band and insert the Full Curve Matrix Band.
- 3) Then use the Composi-Tight® 3D Fusion™ Ring (FX600) to make good contact with wide preps.

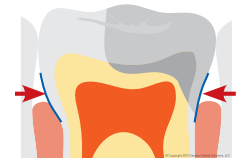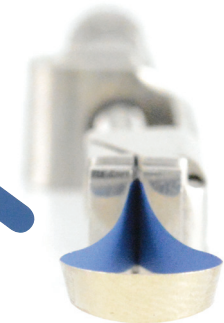






Part No.	Description	Qty.	Price
FX600	Composi-Tight® 3D Fusion™ Ring Wide Prep green	2 rings/pack	£ 185.00
SMT500H10-M	Margin Elevation Band	50 pcs.	£ 45.00

# Composi-Tight® 3D Fusion™

Full Curve Non-Stick Matrices

Fits any tooth shape!

	premolars	premolars with deep cervical preparations	premolars and small molars	molars	molars with deep cervical preparations
Composi-Tight® 3D Fusion™ Bands					
0.038 mm thick					
part number	<b>FX100</b>	<b>FX150</b>	<b>FX175</b>	<b>FX200</b>	<b>FX300</b>
height	4.4 mm	6.0 mm	5.6 mm	6.6 mm	8.7 mm
qty per pack	100 pcs.	60 pcs.	100 pcs.	100 pcs.	60 pcs.

Composi-Tight® 3D Fusion™ Matrix Bands **£ 90.00 per pack**




Full band kit including all sizes with 420 Bands

**FXB01 £ 409.00**

or with 210 bands for

**FXB02 £ 225.00**

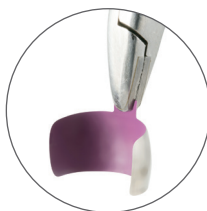

**NEW!**

**Wrap it up** – Full Curve bands wrap further around the tooth, out of the way, making ring placement easier.

**Keep tabs on them** – An occlusal placement Grab-Tab™ gives you excellent control with any placement instrument you choose.

**Go deep or wide** – Improved subgingival extensions and added wrap-around length mean improved results on those more challenging restorations.

**Easy in** – Their advanced curved form improves structural rigidity, making them our easiest-to-place band ever.

**Easy out** – Composi-Tight® 3D Fusion™ Full Curve matrices wouldn't be complete without the Slick Bands™ non-stick coating invented by Garrison.

Item	Description	Price
<b>FXB01</b>	Composi-Tight® 3D Fusion™ Bands Kit 420 pieces (100 pcs. each FX100, FX175 and FX200 and 60 pcs. each. FX150 and FX300)	£ 409.00
<b>BFA-1</b>	Matrix Forceps	£ 127.00

# Composi-Tight® 3D Fusion™

Improved ring placement forceps for simple and precise placement

- Heavy duty stainless steel prevents flexion, making the Composi-Tight® 3D Fusion™ ring placement forceps incredibly durable.
- Composi-Tight® 3D Fusion™ ring placement forceps give you incredible control for the precise placement of super-strong separator rings over even the widest molars regardless of your hand size or strength.
- Works with virtually any ring from any manufacturer.



Part No.	Description	Price
FXP01	Improved heavy duty stainless steel ring placement forceps	£ 80.00

## Step by step for success:

-7-



1. Place the matrix band with Garrison matrix forceps (page 27). Tip: If the teeth are very tight, put the ring on during the preparation.



2. Place the Composi-Tight® 3D Fusion™ wedge with tweezers for a superior gingival seal.




3. Place the Composi-Tight® 3D Fusion™ ring with Garrison ring placement forceps (see left).



# Composi-Tight® 3D Fusion™ Ultra Adaptive Wedges

Combining Soft-Face™ adaptive materials, a firm inner core and advanced mechanical features to produce a wedge that truly works. Every time.

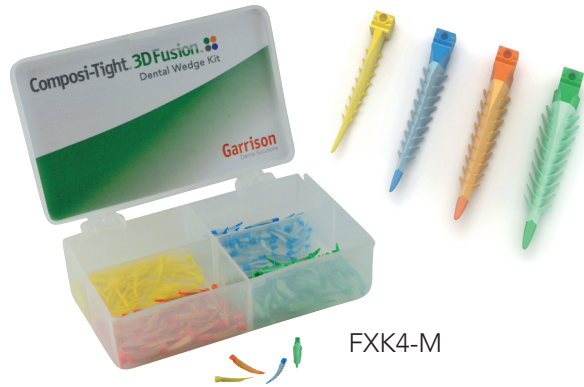

FXK4-M

**Get a grip.**


The universal gripping block gives you absolute control of the wedging process whether you're using a cotton pliers, hemostat or "pin" tweezers. Use what you like – Composi-Tight® 3D Fusion™ has you covered.

**No pain – all gain.**

The upturned ski-like tip and soft radius edges glide over the rubber dam or papilla for trauma free insertion.

**Fins done right.**

Soft retentive fins smoothly fold down during wedge insertion and then spring back when clear of the interproximal space. Wedge back-out is a thing of the past.

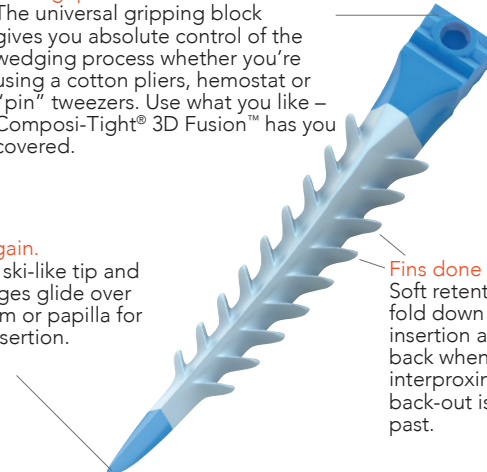

## Composi-Tight® 3D Fusion™ Wedges

Kit	Description	Qty.	Price
FXK4-M	Composi-Tight® 3D Fusion™ Wedge Kit 50 each color	200/pcs.	£ 94.00
FXK4	Composi-Tight® 3D Fusion™ Wedge Kit 100 each color	400/pcs.	£ 180.00

**Refills**

FXYL	X-Small yellow refill	100/pcs.	£ 49.00
FXBL	Small blue refill	100/pcs.	£ 49.00
FXOR	Medium orange refill	100/pcs.	£ 49.00
FXGR	Large green refill	100/pcs.	£ 49.00

**Tough on the inside.**

The firm inner skeleton allows for easy insertion and tooth separation just like a traditional wedge.


**A real softie.**

Composi-Tight® 3D Fusion's Soft-Face™ over-mold allows the wedge to do what no other wedge can truly do – actually adapt to interproximal irregularities.



# Composi-Tight® 3D XR

Upgrade to rings that won't "Spring-Off"!

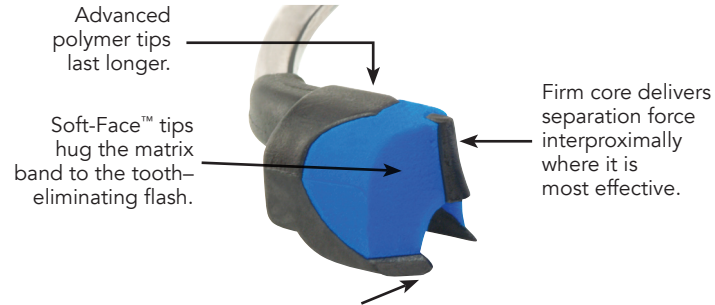

Tip geometry eliminates ring "Spring-Off" on difficult dentition. Restore short or malpositioned teeth and between the canine/bicuspid with confidence. It even grips on top of rubber dam clamps!




**Winner of the 2017 Top Sectional Matrix 5+ and 2013 Editors' Choice Awards**

The Dental Advisor – Top Sectional Matrix System 2013, 2014, 2015, 2016 and 2017 and Editors' Choice 2013.



The Dental Advisor has selected the Composi-Tight® Sectional Matrix System as its Top Sectional Matrix System once again for 2017. The 3D XR ring received the highest possible rating – The Dental Advisor's coveted 5+ rating.

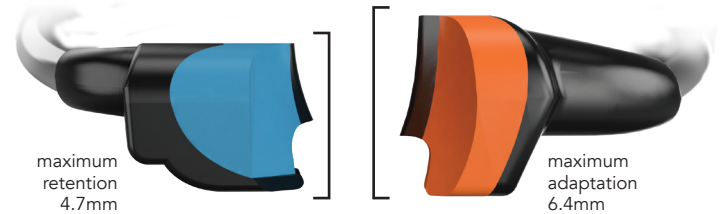

The Composi-Tight® 3D XR Soft-Face™ Sectional Matrix Ring is an ultra-retentive matrix ring designed for use with short or malpositioned teeth, including areas between the canine and first bicuspid.

The orange Soft-Face™ 3D Ring provides maximum ease of placement and adaptation to a wide variety of tooth anatomies while reducing flash.

Please always use rubber dam and our ring placement forceps with all Garrison rings for secure handling.



Ring	Description	Qty.	Price
3DXR	Soft-Face™ Extra Retention Rings (blue)	2/pcs.	£ 180.00
3D500	Soft-Face™ 3D Rings (orange)	2/pcs.	£ 170.00
3D400	Thin Tine G-Rings®	3/pcs.	£ 89.00

# Composi-Tight® 3D XR ☼

Sectional Matrix Systems



3D-KFF-00

**Price: £ 459.00**

## 3D-KFF-00 Kit

- 1 Soft-Face™ 3DXR Ring for an extremely secure grip (blue),
- 2 Soft-Face™ 3D Rings for perfect adaptation (orange),
- 100 Composi-Tight® 3D Fusion™ Bands, assorted,
- 100 Composi-Tight® 3D Fusion™ Ultra Adaptive Wedges, assorted
- 1 Ring Placement Forceps

All kits come with a 60-day-money-back-guarantee. Please register your kit today by phone 0800 011 2738, email to [info@garrisdental.net](mailto:info@garrisdental.net) or send us your order form via fax +49 (0) 2451 971-410.

If you need assistance on the right kit for you please call 0800 011 2738.



3D-KMW-00

**Price: £ 399.00**

## 3D-KMW-00 Kit

- 1 Soft-Face™ 3DXR Ring for an extremely secure grip (blue),
- 2 Soft-Face™ 3D Rings for perfect adaptation (orange),
- 100 Composi-Tight® 3D Matrix Bands
- 100 Wedge Wands®
- 1 Ring Placement Forceps







# Slick Bands™ XR

Easier to place and optimized for wider preps!

	pediatric	premolar	premolar with deep cervical preps	premolars and small molars	molars	molars with deep cervical preps
Slick Bands™ XR Matrices have non-stick coating and Grab-Tabs™ for easier placement						
0.038 mm thick						
part number	N/A	<b>SXR100</b>	<b>SXR150</b>	<b>SXR175</b>	<b>SXR200</b>	<b>SXR300</b>
height		4.6 mm	3.8 mm	5.4 mm	6.4 mm	6.4 mm
qty per pack		100 pcs.	60 pcs.	100 pcs.	100 pcs.	60 pcs.
Slick Bands™ XR Matrix Bands <b>£ 81.00 per pack</b>						

- The non-stick coating greatly reduces adhesion to bonding and composite materials
- Small Grab Tabs™ at the corners facilitate use of the matrices
- Improved shape is optimised for sub-gingival defects
- 5 different sizes, 3-dimensionally contoured and dead-soft

# Composi-Tight3D

	Composi-Tight® 3D Matrix Bands are contoured in three dimensions.					
0.035 mm thick						
part number	<b>M050</b>	<b>M100</b>	<b>M150</b>	<b>M175</b>	<b>M200</b>	<b>M300</b>
height	3.2 mm	4.6 mm	3.8 mm	5.5 mm	6.4 mm	6.4 mm
qty per pack	50 pcs.	100 pcs.	50 pcs.	100 pcs.	100 pcs.	50 pcs.
Composi-Tight® 3D Matrix Bands <b>£ 67.00 per pack</b>						

- 6 different sizes, 3-dimensionally contoured and dead-soft
- 0.035 mm thin for tight contacts
- pre-curved for best anatomical shape

# Composi-Tight® 3D Clear



## Cure-through sectional matrix system

3D Clear allows easy and thorough hardening of deeper proximal boxes and is beneficial for dentists bulk-filling Class II restorations. The clear separator ring has the same tooth hugging soft silicone tip design that virtually eliminates buccal and lingual flash. The bands have tabs on the gingival edge to facilitate placement and are contoured three-dimensionally. In addition, the curvature is enhanced at the marginal ridge to provide improved anatomy.



Set	Description	Price
3D-KCC-01	3D CLEAR Sectional Matrix System 1 Composi-Tight® 3D XR Ring (blue), 2 3D600 Clear Soft-Face™ Rings (translucent), 100 Cure-Through Matrices (30 ea. MC100, MC200; 40 ea. MC175), 100 Cure-Through Wedge Wands® (25 ea. size), 1 Ring Placement Forceps AUMRDF-100, 1 Matrix Forceps BFA-1, 1 Multi-Function Composite Instrument TN009	£ 625.00
3D-KCC-00	3D CLEAR System Kit Includes everything listed above EXCEPT the BFA-1 and TN009	£ 425.00

Composi-Tight® 3D Clear 0.05 mm thick	MC100	MC175	MC200
Most commonly used for:	premolar	bicuspid and small molars	tall molar
Height:	5.0 mm	5.9 mm	6.8 mm
Qty.:	100/pcs.	100/pcs.	100/pcs.

Ring	Description	Qty.	Price
3D600	CLEAR Soft-Face™ (translucent)	2/pcs.	£ 170.00
3DXR	Soft-Face™ Extra Retention Rings (blue)	2/pcs.	£ 180.00
AUMRDF-100	Ring Placement Forceps	1	£ 69.00

Composi-Tight® 3D Clear Matrix Bands

**£ 84.00 per pack**

# Composi-TightGold®

Sectional Matrix System AUK2 - Gold Set:

The second generation Composi-Tight® System still loved by many dentists.



Dr. Martin Able, USA

*"Composi-Tight Gold® is a great matrix, now even better for predictable tight contacts with anatomical contours. This is the benchmark for contemporary dentistry and composite restorations.*

*There are some dental products which I can't practice without. Composi-Tight® Matrices and Wedge Wands® are two such products."*



AUK2

Set	Description	Price
AUK2	Gold System Kit 250 Matrix Bands (100 ea. AU100, AU200; 25 ea. AU150, AU300) 6 G-Rings (3 standard, 3 long) 1 Ring Placement Forceps	£ 425.00



**G-Ringe**

AU400	G-Rings™ Standard Length Tines 3/pcs.	£ 87.00
AU500	G-Rings™ Long Length Tines 3/pcs.	£ 87.00
AUMRDF-100	Ringseparierzange	£ 69.00

Composi-Tight Gold® 0.035 mm thick	AU050	AU100	AU150	AU200	AU300
Most commonly used for:	pediatric	premolars	premolars with deep cervical preparations	molars	molars with deep cervical preparations
height	3.2 mm	4.6 mm	3.8 mm	6.4 mm	6.4 mm
amount per pack	50 pcs.	100 pcs.	50 pcs.	100 pcs.	50 pcs.

Composi-Tight Gold® Matrix Bands **£ 67.00 per pack**

# Composi-Tight®

Sectional Matrix System K102 - Original Set:

Our original system that set the standard for tight, anatomical contacts.

With its naturally pre-formed matrix bands and user-friendly ring system, Composi-Tight® ensures anatomically correct contacts at the natural height. It represented a quantum leap in the production of composite restorations in the posterior region.



Set	Description	Price
K102	Original Kit 225 Matrix Bands (100 ea. B100, B200; 25 ea. B300) 7 G-Rings (4 standard, 3 long) 1 Ring Placement Forceps	£ 375.00



G-Ring™	Description	Quantity	Price
G100	G-Ring™ Standard	3/pcs.	£ 76.00
G200	G-Ring™ Long Length Tines	3/pcs.	£ 76.00
AUMRDF-100 / MRDF-100	Ring Placement Forceps		£ 69.00

Composi-Tight® 0.035 mm thick	B050	B100	B150	B200	B300
Most commonly used for:	pediatric	premolars	premolars with deep cervical preparations	molars	molars with deep cervical preparations
height	3.2 mm	4.6 mm	3.8 mm	6.4 mm	6.4 mm
amount per pack	50 pcs.	100 pcs.	50 pcs.	100 pcs.	50 pcs.

Composi-Tight® Matrix Bands **£ 61.00 per pack**

# BlueView™ VariStrip™

Contoured Anterior Matrices



Variable height  
from 6 -10 mm  
gives you the  
correct height for  
all anterior teeth

The Blue View VariStrip™ is a revolutionary contoured anterior matrix providing the optimum curvature and band height for virtually every anterior restoration. The 0.05 mm thin plastic anatomical strip is tapered from one end to the other. You can place the strip interproximally and then slide it until the tooth height is perfectly matched. The pre-contouring easily recreates the occluso-gingival anatomy, avoiding flat embrasures.

**Curved Form:** Provides the optimum curvature and band height for virtually every anterior restoration.

**Blue Tint:** Provides contrast between matrix and tooth structure without compromising composite polymerization.



Dr. Martin von Sontagh, Österreich

- Avoids incorrect flat embrasures, black triangles and food traps that old-fashioned straight mylars sometimes produce.
- Excellent for Class IVs, diastema closures, etc.




Dr. Gabriele Conte, Italy

## Blue View VariStrip™

Part No.	Description	Qty.	Price
VS02	VariStrip™	50/pcs.	£ 69.00
VS01	VariStrip™	100/pcs.	£ 92.00




# WedgeWands

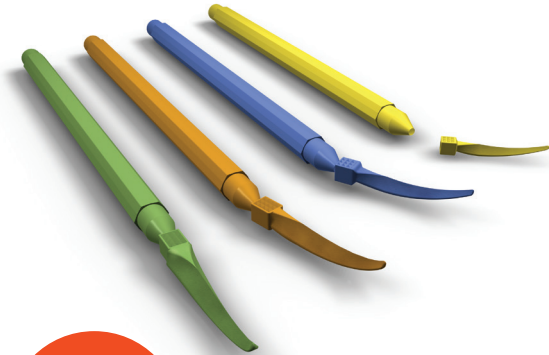

**Stays put.** Back-out is prevented by the textured wedge body and gradual side taper.

**Gentle touch.** The ski-like tip prevents the wedge from digging into the rubber dam or papilla during insertion, and the soft radius edges prevent trauma during insertion and removal.

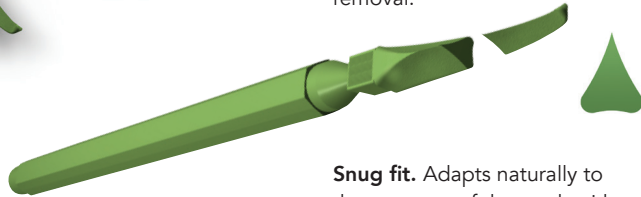

**Snug fit.** Adapts naturally to the contours of the teeth with the pliant plastic material.

**A sure seal.** Every angle and plane is a continuous curve to mimic interproximal anatomy.

## Wedge Wands® Opaque

Set	Description	Qty.	Price
WK4	Wedge Wands™ Kit 100 ea. size	400/pcs.	£ 125.00

**Refills**

SWYL	X-Small yellow refill	100/pcs.	£ 33.00
SWBL	Small blue refill	100/pcs.	£ 33.00
SWOR	Medium orange refill	100/pcs.	£ 33.00
SWGR	Large green refill	100/pcs.	£ 33.00

# WedgeWands

## Cure-Through



WCK4

## Wedge Wands® durchsichtig

Set	Description	Qty.	Price
WCK4	Clear Wedge Wands® Kit 100 ea. size	400/pcs.	£ 125.00

**Refills**

SWCYL	X-Small yellow refill	100/pcs.	£ 33.00
SWCBL	Small blue refill	100/pcs.	£ 33.00
SWCOR	Medium orange refill	100/pcs.	£ 33.00
SWCGR	Large green refill	100/pcs.	£ 33.00

# SoftWedge™

Superior adaptation and seal



The first wooden wedge designed for use with a sectional matrix system

Soft Wedges™ are wooden interproximal wedges that have been optimized for use with modern restorative materials. The ability to compress during insertion, then swell to help fill voids insures that your preparations stay clean and dry – an absolute must for today's bonded restorations! Soft Wedges™ are longer than regular wedges to allow for easy adjustment under the ring. They are made of American bass soft wood for best adaptability and form.



WDK4

## Soft Wedge™

Set	Description	Qty.	Price
WDK4	Soft Wedge™ Kit 100 ea. size	400/pcs.	£ 80.00
<b>Refills</b>			
WDYL	X-Small yellow refill	300/pcs.	£ 59.50
WDBL	Small blue refill	300/pcs.	£ 59.50
WDOR	Medium orange refill	300/pcs.	£ 59.50
WDGR	Large green refill	300/pcs.	£ 59.50

## A+Wedge™



GWAK4

The **A+ Wedge™** controls gingival bleeding making restorations cleaner and quicker. In addition to adapting naturally to the contours of the teeth and mimicking interproximal anatomy, the A+Wedges™ are coated with aluminum sulfate that has an astringent effect on the gingiva. The wedges are designed to prevent back-out and are long enough to allow for proper matrix ring placement.

### A+Wedge™

Set	Description	Qty.	Price
GWAK4	A+ Wedge™ Kit 50 ea. size	200/pcs.	£ 79.00

#### Refills

GWAYL	X-Small yellow refill	100/pcs.	£ 39.50
GWABL	Small blue refill	100/pcs.	£ 39.50
GWAOR	Medium orange refill	100/pcs.	£ 39.50
GWAGR	Large green refill	100/pcs.	£ 39.50

## G-Wedge™

Perfect wedge. Perfect contact.



GWK4

If you prefer traditional wedge placement with either a hemostat or cotton pliers, G-Wedges™ give you the same great shape as Wedge Wands®, just without the applicator wand.

### G-Wedge™

Set	Description	Qty.	Price
GWK4	G-Wedge™ Kit 100 ea. size	400/pcs.	£ 118.00

#### Refills

GSWYL	X-Small yellow refill	100/pcs.	£ 31.00
GSWBL	Small blue refill	100/pcs.	£ 31.00
GSWOR	Medium orange refill	100/pcs.	£ 31.00
GSWGR	Large green refill	100/pcs.	£ 31.00

# SlickBands™ Tofflemire-style


Non-stick technology developed by Garrison: Modern bonding agents can adhere to uncoated matrix bands – Slick Bands™ slide right out.



**SMT500H10-M**

### Margin Elevation Band

A margin elevation matrix will give you the best gingival adaptation and contour for the deep margin areas. The margin elevation matrix is trimmed in height to allow improved adaptation to deep margin areas.




**SMT400D10-M**  
**SMT400H10-M**

### Proprietary Right Curve™ design:

The increased curvature of the all-new Right-Curve™ matrix band produces outstanding gingival seal and reaches over to adjacent teeth easier for broader, deeper contacts. This creates a significantly increased funnel-shaped over to adjacent teeth easier for broader, deeper contacts. This creates a significantly increased funnel-shaped over to adjacent teeth easier for broader, deeper contacts. This creates a significantly increased funnel-shaped over to adjacent teeth easier for broader, deeper contacts.


Reaches over to the adjacent tooth more easily to help you create broader, deeper contacts.

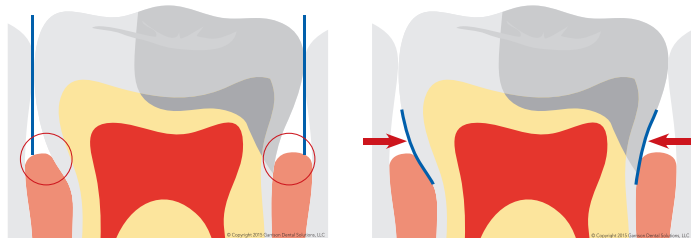

## Slick Bands™

Part No.	Description	Qty.	Price
SMT400D10-M	Right-Curve™ Band dead-soft/green	50/pcs.	£ 40.00
SMT400H10-M	Right-Curve™ Band regular gray	50/pcs.	£ 39.00
SMT500H10-M	Margin Elevation Band	50/pcs.	£ 45.00

# SlickBands™ Tofflemire-style

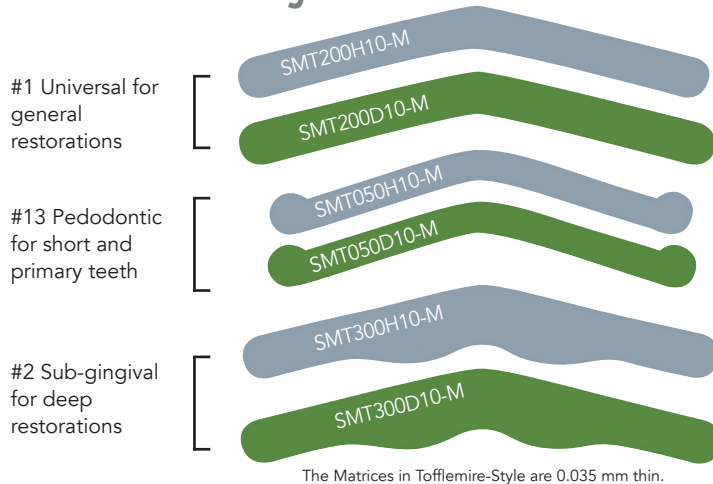

The Matrices in Tofflemire-Style are 0.035 mm thin.

## Slick Bands™ Refills

Part No.	Description	Qty.	Price
<b>Dead-soft bands with non-stick coating (green)</b>			
SMT200D10-M	#1 Universal Tofflemire	50/pcs.	£ 31.00
SMT050D10-M	#13 Pedodontic	50/pcs.	£ 31.00
SMT300D10-M	#2 Sub-gingival	50/pcs.	£ 31.00
<b>Regular Bands with non-stick coating (gray)</b>			
SMT200H10-M	#1 Universal Tofflemire	50/pcs.	£ 27.00
SMT050H10-M	#13 Pedodontic	50/pcs.	£ 27.00
SMT300H10-M	#2 Sub-gingival	50/pcs.	£ 27.00




**Clinical Tip:** Use dead-soft green bands when contact is COMPLETELY broken with adjacent teeth for maximum burnishability. When contact is NOT broken with adjacent teeth, use regular gray bands.

## Slick Bands™ Complete Set


Part No.	Description	Price
SMTK06	Slick Bands™ Tofflemire-style Kit 300 Bands in 9 sizes, including Right Curve™ and Margin Elevation 100 G-Wedges™ in 4 sizes PerForm™ Contact Instruments (1 for small and 1 for large cavities)	£ 259.00

## How PerForm™ works:

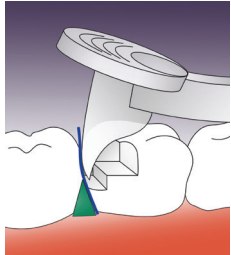


### STEP 1:

Place the first composite increment. Press PerForm into the uncured composite increment and either push (distal, fig. 1)

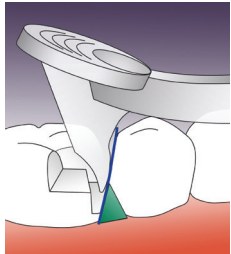


or pull (mesial, fig. 2)

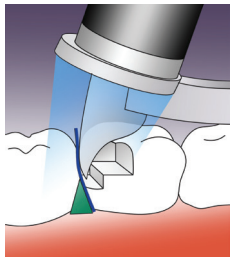

### STEP 2:

Place the curing light directly on the Fresnel lens and cure.

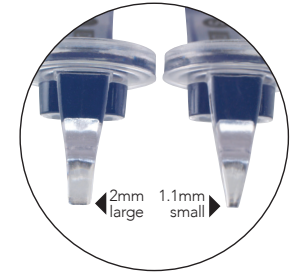

## PerForm™ puts you in control.

- Improves contacts with any matrix system
- Controls the height and width of contacts
- Increases the tightness of contacts
- Unique lens controls curing light diffusion

## PerForm™

Part No.	Description	Qty.	Price
CFK01	PerForm™ Set	1/ea. size	£ 93.00
CF100	1,1 mm Small Instruments	2/pcs.	£ 93.00
CF200	2,0 mm Large Instruments	2/pcs.	£ 93.00

“To ensure a tight contact a bulk fill flowable was placed in one proximal box at a time and the mylar held against the adjacent tooth with a “PerForm” proximal contact instrument.

This ensures that the contact is tight and well-formed while light curing through it.”

Dr. Jack D. Griffin, Jr. USA



## Mini disposable polisher for composite and hybrid ceramics

### Remove Smooth Shine

Garrison's new Rally™ Single Patient Mini-Polishers provide clinicians with the points and cups necessary to successfully finish and polish composite restorations to a lustrous, durable shine.

Color coded and conveniently organized, the Rally™ Mini Polishers are available in an assortment kit and as a selection of refills.

### Rally™


Part No.	Description	Qty.	Price
FPCC010-X	Blue coarse grit point	a) 30 pcs.	£ 55.00
FPCC040-X	Blue coarse grit cup	b) 30 pcs.	£ 55.00
FPCM010-X	Red medium grit point	c) 30 pcs.	£ 55.00
FPCM040-X	Red medium grit cup	d) 30 pcs.	£ 55.00
FPCF010-X	Gray fine grit point	e) 15 pcs.	£ 55.00
FPCF040-X	Gray fine grit cup	f) 15 pcs.	£ 55.00



### FPCK01-X Rally™ Starter Set

24 Polishers (12x cups, fine grit and 5x each, medium and coarse grit; 12x point, 2x fine grit and 5x each, medium and coarse grit)



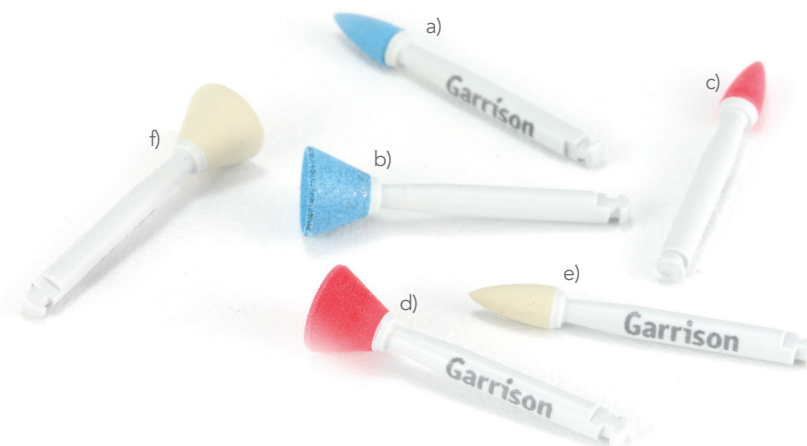







**£ 55.00**





## FitStrip™ Interproximal Finishing and Contouring System
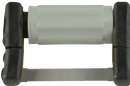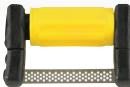
### FitStrip™ Refills

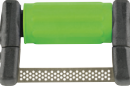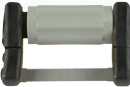FitStrip™ single sizes contained in starter set					
<b>single-sided strips</b>					
item	<b>FPSXFSS</b>	<b>FPSFSS</b>	<b>FPSMSS</b>	<b>FPSCSS</b>	<b>FPSSAW</b>
thickness	0.08 mm	0.10 mm	0.13 mm	0.18 mm	0.05 mm
	<b>super fine / yellow</b> 15 µm	<b>fine / red</b> 30 µm	<b>medium / blue</b> 46 µm	<b>coarse / green</b> 90 µm	<b>serrated</b>
<b>double-sided strips</b>					
item	<b>FPSXFDS</b>	<b>FPSFDS</b>	<b>FPSMDS</b>	<b>FPSCDS</b>	<b>FPSSAW</b>
thickness	0.11 mm	0.15 mm	0.21 mm	0.30 mm	0.05 mm

All sizes can be ordered separately. Please call us for further information.

FitStrip™ Refills contain 10 strips of one size each and 2 interchangeable handles.

Each FitStrip™ refill pack

**Price: £ 149.00**





## FitStrip™ Interproximal Finishing and Contouring System

- Serrated strips and diamond abrasive strips with interchangeable handles for safe and easy use in the patient's mouth.
- A simple twist of the color-coded barrel adjusts the strip from straight to curved and is perfect for:
  - *interproximal contouring of composit fillings (curved form)*
  - *contact reduction before or after the restoration (straight form)*
- Best solution for IPR (interproximal reduction) with orthodontic treatments such as Clear Correct™, Invisalign™, Six Month Smiles™.
- Multiple cleaning and sterilisation cycles



FPSK01

**Price: £ 149.00**

### The FPSK01 Set contains:

- 2 handles
- 2 serrated FitStrips™
- 4 single-sided FitStrips™ of each grit
- 4 double-sided FitStrips™ of each grit



# Making adjustments eZr. for you!

The correct processing of zirconia has been a controversial issue for a long time, due to the risk of microfracturing. Finishing with the appropriate instruments is now possible and produces long term stability of the restoration.

The eZr™ Polishing System provides a processing solution that not only produces a high gloss finish on zirconia, but increases the physical stability through polishing. Highly polished stable zirconia surfaces are now attainable. The polishers have a long life, and in combination with a high material removal rate, allow for an economic and fast processing system to get to glossy in seconds, without polishing paste.

- Enables easy and safe adjustment for zirconia and lithium disilicate.
- Proprietary diamond impregnated polisher reduces possibility of chipping and microfracturing.
- Provides an unparalleled smoothly polished occlusal surface, eliminating abrasiveness to opposing dentition.
- Prevents over-heating that could lead to premature failure.
- Eliminates the need for polishing paste, reducing overall cost per procedure.

	Item	Description	Grit	Price
	FPZC030-X	Blue coarse grit disc gross removal, 5/pack	Gross removal	£ 130.00
	FPZC010-X	Blue coarse grit flat end taper gross removal, 5/pack	Gross removal	£ 98.00
	FPZM010-X	Green medium grit small point polisher, 5/pack	Smoothing and Pre-Polishing	£ 41.00
	FPZM020-X	Green medium grit large point polisher, 5/pack	Smoothing and Pre-Polishing	£ 61.00
	FPZM030-X	Green medium grit disc polisher, 5/pack	Smoothing and Pre-Polishing	£ 61.00
	FPZM040-X	Green medium grit cup polisher, 5/pack	Smoothing and Pre-Polishing	£ 61.00
	FPZF010-X	Orange fine grit small point polisher, 5/pack	High Gloss Polishing	£ 41.00
	FPZF020-X	Orange fine grit large point polisher, 5/pack	High Gloss Polishing	£ 61.00
	FPZF030-X	Orange fine grit disc polisher, 5/pack	High Gloss Polishing	£ 61.00
	FPZF040-X	Orange fine grit cup polisher, 5/pack	High Gloss Polishing	£ 61.00



eZr™ polishers  
enable clinicians to easily and  
safely adjust and polish both  
**zirconia** and **lithium disilicate**.



- 1 Start by gross adjusting and contouring using the blue coarse grit grinders. Use the flat end taper for occlusal/labial/lingual/buccal surfaces and the disc for interproximal surfaces.



- 2 Use the green medium grit polishers for smoothing and pre-polishing. Use the point for the occlusal grooves, the cup for labial/lingual/buccal, and the disc for interproximal surfaces.



- 3 Use the orange fine grit polishers for high gloss polishing. Use the point for the occlusal grooves, the cup for labial/lingual/buccal, and the disc for interproximal surfaces.



FPZK01-X - eZr™ Kit

## FPZK01-X – eZr™ Kit

Kit with 2 grinders and 6 polishers

2 grinders for gross removal (blue)

3 polishers for smoothing and pre-polishing (green)

3 polishers for high gloss polishing (orange)

£ 165.00

# Composi-Tight<sup>®</sup> Matrix Forceps

Precise Placement, Unbeatable Grip.

**Precise Placement:** The tips have a 75-degree angle with an occlusal-lingual orientation, providing a more appropriate placement angle than cotton forceps. The petite, carbide tips allow the matrix to be grasped at its upper edge to prevent distortion of the interproximal area of the band.

**Unbeatable Grip:** The tungsten carbide tips provide exceptional grip and tactile control for tough access cases.



Only for Matrix Bands.

Matrix Forceps


BFA-1

Angled tip for occlusal placement

£ 127.00

# Universal Forceps

Holds all your rubber dam clamps and sectional matrix retainer rings.

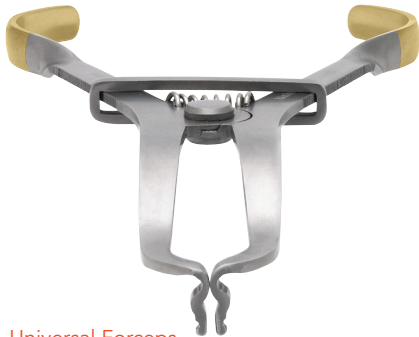

UNVF-100



Universal Forceps

UNVF-100 Universal Forceps

£ 80.00

# Ring Placement Forceps

Holds all your sectional matrix system rings.

Updated grip to easily spread rings with less hand fatigue, even over large molars.

Can be used on all popular sectional matrix rings.

The ring tips surround the ring: no wobbling or slipping.

AUMRDF-100




Ring Placement Forceps

AUMRDF-100 Ring Placement Forceps £ 69.00

# Composite Instruments Nonstick titanium-nitride coated

Our multi-function instruments have double condenser configurations and carving blades on opposite ends. Now you can place, pack, shape and carve your way to perfectly contoured composite restorations without ever having to switch instruments!

- Large-diameter, lightweight handles for improved ergonomics.
- Provides non-stick surface for placement and shaping of composites.
- 40% harder than stainless steel for a non-porous, non-stick surface.
- Withstands diverse cleaning/sterilization methods, including ultrasonic cleaning.

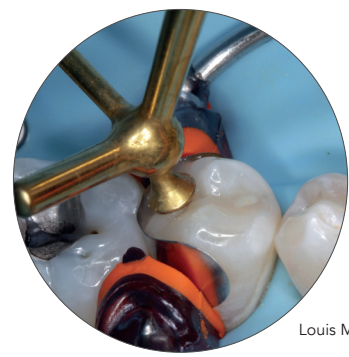

TN009,  
Louis Mackenzie,  
UK



TN008

5-in-1

Part No.	Description	Price
TN008	5-in-1 Multi-Function Instrument 2 condensers, ball burnisher, blade, acorn	£ 99.00



TN009

5-in-1

Part No.	Description	Price
TN009	5-in-1 Multi-Function Instrument 2 blades, 2 condensers, acorn	£ 99.00

# Garrison Instruments

## Double Cord Packer



TN010

Part No.	Description	Price
TN010	Double Cord Packer 2 serrated blades, 2 non-serrated blades	£ 75.00

A simple twist of the instrument allows you to maneuver around the tooth without ever leaving your field of view. Perfect for use with loupes and microscopes.

# Composite Instruments

Nonstick titanium-nitride coated

## Composite instruments with non-stick titanium nitride coating

TN001



Part No.	Description	Price
TN001	Medium Blades	£ 40.00

- Placement of composite material
- Gingival retraction to sculpt under free margins
- Burnishing matrix bands

TN002



TN002	Medium Blade with Medium Plugger	£ 40.00
-------	----------------------------------	---------

- Contouring small Class I, II, V
- Packing composite material into conservative preparations
- Burnishing matrix bands

# CompositeInstruments Nonstick titanium-nitride coated

## Composite instruments with non-stick titanium nitride coating

Part No.	Description	Price
TN003	Medium Blade with Large Plugger	£ 46.00

- Contouring large Class I, II, V
- Packing composite material into large preparations
- Convex plugger tip minimizes pullback of composite material

TN003



TN004	Extra Thin Blades	£ 46.00
-------	-------------------	---------

- Reach under margins to hone and sculpt
- Sculpting interproximally
- Packing cord in gingival crevice

TN004



TN005	Long Extra Thin Blades	£ 46.00
-------	------------------------	---------

- Allows additional access for hard-to-reach
- Posterior contouring and shaping
- Sculpting interproximally
- Packing cord in gingival crevice

TN005



TN006	Small/Medium Multi-Function Plugger	£ 55.00
-------	-------------------------------------	---------

- Placement of composite material in smaller Class II and III
- Packing material toward gingival seats
- Carving occlusal anatomy with acorn carver

TN006



## Key Opinion Leaders working with Garrison products



Dr. Martin von Sontagh, Hard, Austria

*"I use the Composi-Tight® 3D XR System in almost all Class II restorations, as it is a simple and reliable system. Thanks to the quick and easy placement of the rings, you can easily get tight contact points. As the rings have a very tight fitting, composite excess is minimized and the finishing time is reduced. If the*

*system once is in place, there is no jumping or slipping anymore. The results are anatomically wellshaped contact points and functional restorations."*



Dr. Walter Denner, Fulda, Germany

*"The Composi-Tight® 3D sectional matrix system enables the production of tight, anatomically contoured approximal contacts in an easy and reproducible manner. The sectional matrices with colour on the inside also give a very good contrast and facilitate the layering of the composite materials. The strong 3D ring ensures*

*that the matrix adapts well to the remaining tooth and has space for an interdental wedge so that flash is easily reduced. For tight areas the smaller ring can be used so that almost 100% of all class II restorations in the lateral tooth area can be covered with this system."*



Louis Mackenzie, Birmingham, UK

*"The revolutionary design of the Garrison TN009 multi-function composite instrument makes it ideal for precise placement and shaping of direct composite restorations. With five non-stick tips it allow fast, efficient, predictable shaping of occlusal anatomy. If you only buy one specialised composite*

*instrument the TN009 could not be a better investment."*





Dr. Carlos Fernández Villares, Madrid, Spain

*"The matrix bands and rings from Garrison enable us to achieve restorations that are anatomically identical to the natural tooth, with contact points and areas that allow for adequate function and esthetics."*



Dr. Rodolphe Zunzarren, Biarritz, France

*"The VariStrips™ from Garrison bring several important characteristics together which make them perfect for anterior restorations:*

- *The blue colour allows a perfect visibility without interference of the photopolymerisation*
- *Thanks to different sizes in one band, the VariStrips™ are easily adjustable for different heights of the cavity or different restoration angles*

- *The curved form allows an anatomical restoration of the proximal teeth form and of the contact points*

*Being both rigid and thin, the VariStrips™ are easy to insert and thanks to the minimal thickness of the bands, they allow a perfect reconstruction of the contact point."*



Dr. Matthew Holyoak, Lytham St. Annes, UK

*"The 3D XR ring from Garrison Dental Solutions is a significant improvement on all previous rings; it is truly universal and I have not found any Class II restoration that it will not fit. The 3D XR has not flown across the room once! When used with the sectional matrices, and Wedge Wands® (or A+ Wedges™), the system helps provide a tight*

*anatomical contact area and highly predictable posterior composite restorations every time."*

# Garrison

Dental Solutions

**NEW! Now order conveniently through our online shop at [www.garrisdental.com](http://www.garrisdental.com)**

Delivery via your desired dental depot. Registration take only a few steps. If you have any questions, please do not hesitate to contact us by phone or email.



You can reach us by phone between 8:00 am and 5:00 pm Monday through Friday. If we are not personally available for you, just leave a message with your name and phone number and we will call you back as soon as possible. Tel. 0800 011 2738