HELIX TEST PRO

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Instructions for use (english/Rev 07/19)

REF 243920 (Helix Test Pro)
REF 243921 (Helix Test Pro Refill)

Batch control test according to EN 867-5 as batch control agent for B-class autoclave programs.

Application

- **When** Due to process safety, we recommend the application for ANY batch of a B-program.
- **How:** Make sure that the Helix Test PRO is dry and intact before use. Remove the black screw cap (Fig.1). Slide the centrally folded indicator strip with the color area facing in (Fig.4) and the fold (crease) first into the holder provided for this purpose. Make sure that the indicator strip does not slip out of the holder. Screw the cap back on.
- **Positioning:** Place the prepared Helix Test PRO in the autoclave at the location specified during validation (or according to the device manufacturer's specifications). If both are not available, we recommend positioning close to the door.
- **Evaluation:** When the B-program is finished, remove the Helix Test PRO from the chamber. **CAUTION:** may be hot – remove the cap. Remove the indicator strips from the holder. The colour change must be uniform from blue to pink. The results of the controls must be documented in writing or digitally and form the basis for the risk assessment for extended re-validation cycles. In the event of unsuccessful results, we recommend that the measures taken to ensure or restore sterilization quality also be documented. If required, the indicators can be easily archived using the self-adhesive reverse side.

Maintenance and Care

With each refill pack of the indicators (Refill: REF 243921) you receive a new O-ring for the thread. The change is easy and is done by unscrewing the indicator holder from the black cap. This must be replaced after 250 cycles to avoid leaks and incorrect measurements. Only original indicators and O-rings must be used and the statement of the test results must not be falsified.

Storage

The indicators must be stored cool and protected from light before and after use.

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