METAL-BITE®

Universal precision registration material



2014
REALITY
Four Star Award







www.r-dental.com

Product information





The Editors' Choice:

METAL-BITE ®

"My new favorite bite registration!"

"Easy to dispense."

"Sets incredibly fast."

"Not brittle!"

"Easy to use."

"Trims easily."

"No taste at all!

www.dentaladvisor.com





METAL-BITE®

High quality product for universal and CAD/CAM precision registrations

Introduction

Indications

Performance

parameter


Early in 2000 the classical dark grey **METAL-BITE** ® was introduced as the first universal fast setting and scannable automixing vinyl polysiloxane. It may be used for conventional intraoral registrations (static occlusion registrate in hicp) and for chairside and labside powderfree scanning with universal CAD/CAM systems. **METAL-BITE** ® is recommended by SIRONA for CEREC worldwide. Until now the universal **METAL-BITE** ® is the standard in the dental CAD/CAM registration. The scan characteristics of **METAL-BITE** ® products provide optimal data for totally faultless antagonist and "natural" design suggestions for restorations.

Over time the conventional indications have been extended: As a result of innovative applications **METAL-BITE** ® is not only used for static but also for dynamic registrations and because of its radiopacity **METAL-BITE** ® is used additionally for manufacturing surgical guides for implantology and for bite plates of face bow registrations. The registrations are rigid and easy to trim.

The indications of **METAL-BITE** ® are as follows:

- · universal registrations and static hicp-registrations
- · classical face-bow registrations (according to Prof. Dr. A. Gutowski, Germany)
- for CMD applications according to Dr. A. Könecke and PD Dr. M. O. Ahlers (literature on request).
- antagonist registrations and powderfree scanning with universal CAD/CAM systems
- dynamic registrations according to the FGP-Method (compare Dr. Günter Fritzsche, Efficient dynamic registration technique using METAL-BITE ® presented with a CEREC 3D partial crown in: International Journal of Computerized Dentistry, IJCD, Volume 10, Number 1, 2007, QV)

METAL-BITE $^{\circledR}$ surpasses the requirements of the standard DIN 13903 (Dentistry - Jaw relation recording material). The performance parameters are shown as follows:

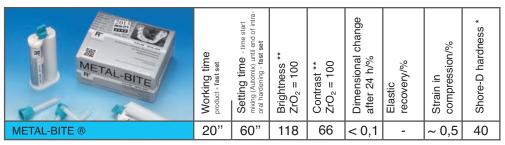

^{* 24} h-Value according to DIN 53505 (Testing of rubber - Shore-A and Shore-D hardness test), ** Relative values, internal

Benefits at a glance:

METAL-BITE ® offers you

- · a universal and scannable precision registration material for chairside and labside indications
- · a comfortable working time for a stress free registration
- an excellent thixotropic behaviour (excellent standing and flowable character)
- an excellent snap-set (this is a temporally steep, sigmoidal (S-form) phase history during the polymerisation phase)
- · a precise detail reproduction
- an excellent brightness, contrast and great scanning detail without powder
- a high Shore-D hardness to prevent deformation
- an excellent cutting, contouring, easy trimming and easy withdrawal from the mouth.

METAL-BITE ® should be stored below 25 °C (78 °F) and horizontally.

Order information:

Order information

Description :Contents:Order-No.:R-SI-LINE ® METAL-BITE TM (dark grey)2 cartridges à 50 ml + 12 micMBK1074

R-SI-LINE and METAL-BITE are registered trademarks of R-dental Dentalerzeugnisse GmbH. SIRONA and CEREC are not registered trademarks of R-dental Dentalerzeugnisse GmbH. Product changes and errors excepted.

© R-dental Dentalerzeugnisse GmbH 2000-2014. All rights reserved.

Further products, information and catalogue:

R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg • Germany

Phone: +49 40-22757617 • Fax: +49 40-22757618 • e-mail: info@r-dental.com • Web: www.r-dental.com

Benefits

Contact