



POLISHING WITH

# ***TWIST™ DIA for Composite***

A natural gloss in only 2 steps

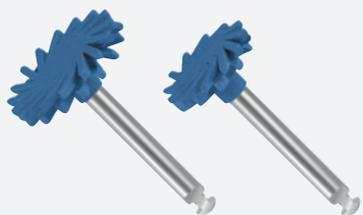

● **MEDIUM**  
PRE-POLISHING

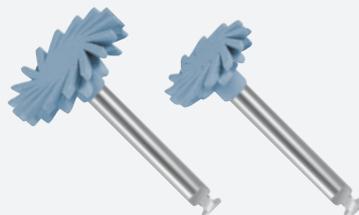

● **FINE**  
HIGH-SHINE POLISHING


# THE EASIEST WAY TO CREATE NATURAL GLOSS

TWIST™ DIA is indicated for all sorts of composites. The 'pre-polisher' creates the final form from any angle imaginable. With the 'high-shine polisher', you can finish with a natural high-gloss result in seconds. TWIST™ DIA is autoclavable and thus reusable. The combination of flexible components and diamond particles give you an unprecedented high-gloss result with each use!

## BENEFITS

- Easy polishing procedure thanks to just 2 polisher
  - Pre-polisher to smooth the surface
  - High-shine polisher to create natural gloss
- Sterilizable and reusable
- Economic efficiency due to high durability and reusability
- 100% diamond grains inside - no substitutes or cheaper abrasive particles

SCAN TO VIEW  
PRODUCT IFUs

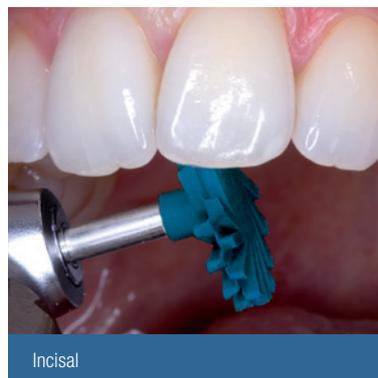

### RECOMMENDED AND MAXIMUM ROTATION SPEEDS

Recommended speed: 3.000 – 8.000 rpm for Medium/Fine

Maximum speed: 20.000 rpm

## TWIST™ DIA for Composite

Composite finishing and polishing system

#1201-IP 14mm Introductory kit (6pcs)

#1202-IP 14mm Pre-polishing Refill (10pcs)

#1203-IP 14mm High-shine polishing Refill (10pcs)

#1204-IP 14mm Trial (2pcs)

#1211-IP 10mm Introductory kit (6pcs)

#1212-IP 10mm Pre-polishing Refill (10pcs)

#1213-IP 10mm High-shine polishing Refill (10pcs)

#1214-IP 10mm Trial (2pcs)

#1221-IP Pre-polishing combination kit (10pcs)

#1222-IP High-shine polishing combination kit (10pcs)

## POLISH ZIRCONIA WITH TWIST™ DIA

The innovative shape with flexible polishing spirals offers various application benefits to the dentist for excellent polishing results. Find out more: [kuraraynoritake.eu/twist-dia](http://kuraraynoritake.eu/twist-dia)

Before using this product, be sure to read the Instructions for Use of the product.

The specifications and appearance of the product are subject to change without notice. Printed color can be slightly different from actual color.

### Distributed by:

Kuraray Europe GmbH, BU Medical Products, Philipp-Reis-Str. 4, 65795 Hattersheim am Main, Germany, +49 (0)69-30 535 835, [dental.eu@kuraray.com](mailto:dental.eu@kuraray.com), [www.kuraraynoritake.eu](http://www.kuraraynoritake.eu)



06/2021