SOPIRA[®] Ject N

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2-3

EN - english
Instructions for use

4-5

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6-7

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8-9

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10-11

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Instructions for use

Cartridge syringe for dental use with 3 aspiration hooks and locking device (up to max. 1.8 ml - metric thread)

Use:

The cartridge syringe can be used for all standardised glass anaesthesia cartridge up to max. 1.8 ml with a hole plug.

CAUTION: The use of solid plugs is not possible, as otherwise the aspiration hooks could be damaged.

IMPORTANT: Sterilise the cartridge syringe before use in humans.

The Sopira Ject N syringe is sold non-sterile. Before first use, carry out a complete processing according to the processing instructions (see the Sopira Ject N instructions for use and processing supplied or on the carlmartin.de website under Downloads "QSA 313").

Prepare for injection by inserting the 1.8 ml anaesthesia cartridge with hole plug:

First, the plunger is fully retracted, overcoming a spring resistance at the end. The front part of the syringe with the anaesthesia cartridge holder can then be swung down at a right angle.

The anaesthesia cartridge is inserted with the aluminium cap facing forwards. The cartridge holder is now swung back again, whereby the plunger must be pulled back against the spring resistance.


The plunger is pushed forward into the recess of the hole plug. The necessary anchoring is then created by turning the knurled screw, which is located directly in front of the plunger ring, clockwise. It is important to ensure that the rotary mechanism is fitted with a locking mechanism that engages in the end position after the aspiration hooks have been turned clockwise and swung out. The aspiration hooks* are swung in by turning the knurled screw anti-clockwise.

After inserting the cartridge (anaesthetic), the disposable needle is carefully inserted and screwed onto the thread. After injecting a few drops of anaesthetic, the syringe is ready for use.

***Note:**

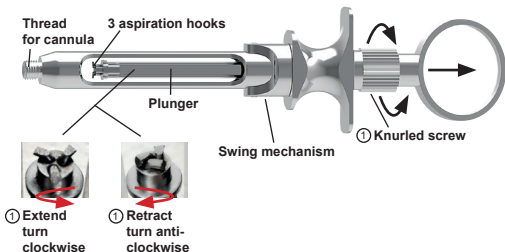The aspiration hooks must be retracted when inserting and removing the anaesthesia cartridge to prevent damage.

The turning mechanism inside the knurled screw must be oiled regularly.

First insert the cartridge and then screw on the injection needle.

Add 1-2 drops of oil to the turning mechanism.

After use on a patient, the cartridge syringe must be cleaned, disinfected and sterilised before further use in accordance with our processing instructions (QSA313).

Care must be taken in particular here to ensure that any residue of anaesthetic agent inside the cylinder is immediately removed after the treatment by manual cleaning with subsequent visual inspection. (Anaesthetic can attack the metal surface)

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Kulzer GmbH
Carl Martin GmbH