KULZER VIVIDA

WHITENING

6% HYDROGEN PEROXIDE SUPERIOR
OFFICE PROPHY PEN

INSTRUCTIONS FOR USE

KULZER.COM/
VIVIDA-PROPHY

A BEAUTIFUL SMILE
CAN BE SO EASY

COSMETIC PRODUCT – FOR PATIENTS UNDER DENTAL SUPERVISION AND FOR DENTAL PROFESSIONALS

INTENDED PURPOSE

Kulzer Vivida Office Prophy Pen is a single use in-office product recommended to be applied by dental professionals right after a professional dental cleaning so that the patient becomes familiar with the sensation of tooth whitening.

The 6% HPS gel is used as a hydrogen peroxide-containing bleaching agent in Kulzer Vivida Office Prophy Pen. It is provided encapsulated as HPS Hydrogen Peroxide Superior in order to obtain a sufficiently stable product. The HPS Hydrogen Peroxide Superior dissolves quickly in oxygen radicals and water. During the bleaching process, the resulting free radicals attack the double carbon compounds in organic molecules, which in most cases are the cause of tooth discolouration.

INDICATIONS

For correcting and whitening tooth colour.

CONTRAINDICATIONS

The use is contraindicated for

- pregnant and breastfeeding women
- children under the age of 18
- known hypersensitivity or allergies to mint flavour or any of the other ingredients
- those currently treated for a serious illness

POTENTIAL SIDE EFFECTS

Hypersensitivity to the product or its ingredients cannot be ruled out in individual cases.

CONTENTS

12 x Kulzer Vivida Prophy Pen 1.4 ml

Active Ingredient:

- 6% Hydrogen Peroxide

Composition:

PEG200, water, hydrogen peroxide, poloxamer, T-butyl alcohol, etidronic acid, mint flavouring, potassium hydroxide, EDTA, sodium stannate

WARNINGS:

- Contains hydrogen peroxide.
- Avoid contact with eyes. Rinse immediately if product comes into contact with them.
- Keep out of reach of children
- Avoid exposure of known sensitive teeth.
- Do not eat, drink or smoke while wearing the tray.
- Harmful when swallowed.

COLOUR

Transparent

PRODUCT APPLICATION

1 PRE-TREATMENT

After you have performed a professional dental cleaning, advise your patient on the option of whitening his/her teeth with a 6% Hydrogen Peroxide Gel.

- Use on patients under the age of 18 is not permitted.
- Inform the patient fully about the treatment procedure and its risks. The patient should not have any unanswered questions in this regard.
- Assess the patient's oral health and clarify whether the patient's expectations are realistic. Inform the patient that in a small percentage of patients the teeth will not lighten significantly.
- Check the patient's dental status and oral health to assess whether pre-treatment is indicated. All restorations should be intact or sealed with suitable temporary materials.

- Inform the patient that temporary restorations may change colour during the bleaching process, while composite or polycarbonate materials, metals and ceramic restorations will not be affected. Existing restorations may need to be replaced after bleaching, as only natural teeth will become whiter.
- For patients requiring aesthetic or adhesive restorations, wait at least 2 weeks after the last whitening session to allow the colour to stabilise.
- Advise the patient of possible tooth sensitivity and irritated gums during treatment. The gums that come into contact with whitening gel may turn red or white, but will return to their normal colour after a few days.
- For a full whitening treatment and visible results, application would have to be repeated for at least 2 weeks. In the case that the patient decides to continue the treatment, it is recommended to switch to Kulzer Vivida Home Kit for use at home.

2 MAIN TREATMENT

- Carefully turn the rear part of the Kulzer Vivida Office Prophylaxis Pen clockwise to move the whitening gel into the brush area. Be careful not to apply too much material to the brush at once. Then apply a thin layer of Kulzer Vivida Office Prophylaxis Pen gel onto the teeth. The amount of whitening gel should be applied in moderation.
- Be careful and make sure that no gum or other soft tissue come into contact with the whitening gel.
- In case of hypersensitivity, the duration of the application should be shortened or interrupted.
- After 10 to 15 minutes remove the gel by suction
- You can repeat the application and removal steps up to 2 times.
- Rinse the teeth with water and suction.

PRODUCT APPLICATION

3 POST-TREATMENT

- You can recommend to the patient to use Kulzer Vivida Tooth Conditioner after the whitening process.
- Advise the patient not to eat, drink or smoke for at least 30 minutes after the treatment. To achieve the best possible result, the patient should avoid colouring foods and drinks such as coffee, tea, red wine and berries for at least 24 hours after treatment.

INFORMATION

GENERAL INFORMATION

Please carefully observe the following safety instructions and all processing instructions in other sections. This product should only be used in accordance with the instructions for use. Any use not in accordance with these instructions is at the discretion and sole responsibility of the dental professional.

SAFETY INSTRUCTIONS

Eyes and skin may become irritated. Avoid contact with skin. Sensitization through skin contact possible. If the product comes into contact with the skin, wash thoroughly with water immediately. Avoid contact with eyes. In case of contact with eyes, rinse with water for several minutes. Consult an ophthalmologist if symptoms persist. Keep out of the reach of children.

STORAGE CONDITION

- Do not use after the expiry date
- Single use
- Store at temperatures between 4 °C and 25 °C (39 °F and 77 °F).
- Protect from sunlight.
- Store in a dry place.

DISPOSAL INFORMATION

Dispose of in accordance with local regulations.

ADDITIONAL INFORMATION

When giving feedback on the product, please always state the batch designation LOT and article number REF. Article number and batch designation: See information on syringe/tube/packaging.

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READY TO BRIGHTEN
YOUR PATIENTS' SMILES?

VISIT
[KULZER.COM/VIVIDA-HOME](https://www.kulzer.com/vivida-home)

- Storage at room temperature
- Gentle treatment in only 15–30 min
- High quality packaging

 Cavex Holland BV
Fustweg 5
2031 CJ Haarlem
(The Netherlands)

 Kulzer GmbH
Leipziger Straße 2
63450 Hanau (Germany)
Kulzer.com

KULZER
MITSUI CHEMICALS GROUP

Explanation of symbols on labelling

	Reorder no.
	Manufacturer
	Distributor
	Batch code
	Use-by date
	Period after opening
	Consult instructions for use
	Read more
	Storage temperature
	Keep away from sunlight
	Keep dry